
QUICKSTART GUIDE

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BIONOVA®
Hyper



Terragene®
Rev. 1 | June 2021

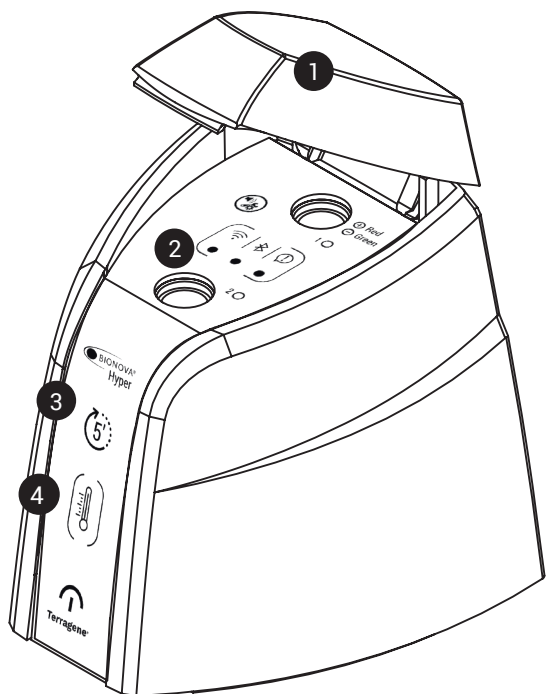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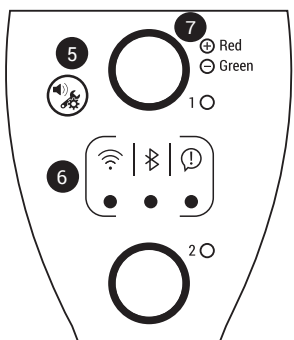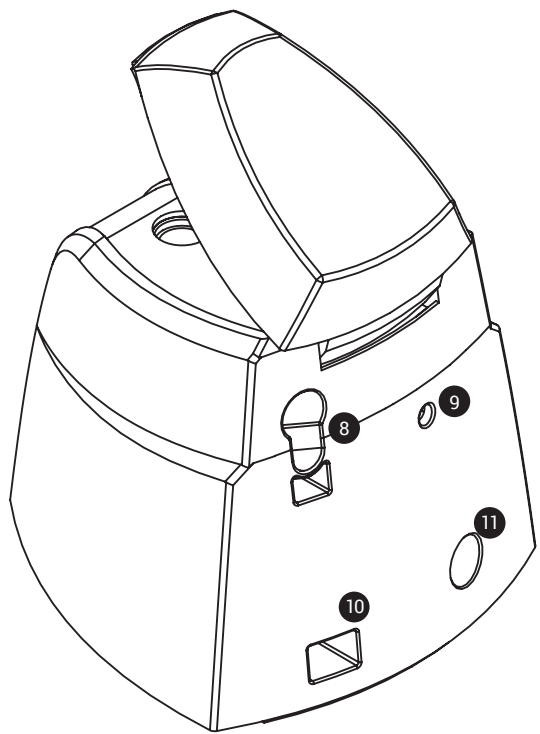EN Instructions for use

Setting Up your Auto-reader

Place the Bionova® Hyper Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto-reader

- 1 | Power on your Hyper Auto-reader by connecting the power supply's AC plug to mains and then connect the plug at the other end of power supply to the rear of the Auto-reader. (See image 1)
- 2 | Wait until the Temperature Progress & Stability Indicator stops blinking. The Hyper logo will turn on when the Auto-reader is ready to process a sample. Place a SCBI in a readout position. The incubation will start automatically. (See image 2)
- 3 | Wait until the incubation is complete. The position status indicator will turn green to indicate a negative result, or red to indicate a positive result. If a positive result is found, press the alarm/configuration button to silence the alarm. Do not remove the SCBI until a result is informed. (See image 3)
- 4 | Run Bionova® Traceability Software or the Bionova® Wireless Assistant mobile app to check the incubation result e-ticket.

Disposal: Discard the SCBIs following your Country disposal directives.

Connectivity

To connect your Hyper Auto-reader to a PC via USB, connect one end of the included USB cable to the rear of the incubator and the other end to an USB port on your PC. Wi-Fi connectivity is also available and can be Set-up using the Bionova® Wireless Assistant. Connecting the Auto-reader to a PC (by USB, or Wi-Fi) allows using Bionova® Traceability Software to save, manage, and print e-tickets reports. Connecting the Auto-reader to an Android device (by Wi-Fi or Bluetooth®) allows using the App to check the status of the device, access e-tickets, and more. Check the User Manual for more information.

Product Compatibility and Incubation programs

Bionova® Hyper Auto-reader Incubation Programs

The following incubation programs are available for your Bionova® HyperAuto-reader.

Temperature	Time	Incubation Program
60 °C	Hyper (5 min)	Hyper at 60 °C

! Follow each Indicator's instructions for use before its incubation.

Bionova® Hyper Auto-reader compatible indicators

We develop new Indicators for extending the features of our Auto-readers regularly. Please visit www.terragine.com/HYPER to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible Indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

Troubleshooting

Refer to the user manual for a detailed troubleshooting routine. Please visit www.terragine.com/HYPER or scan the QR code to access the Auto-reader user manual.

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