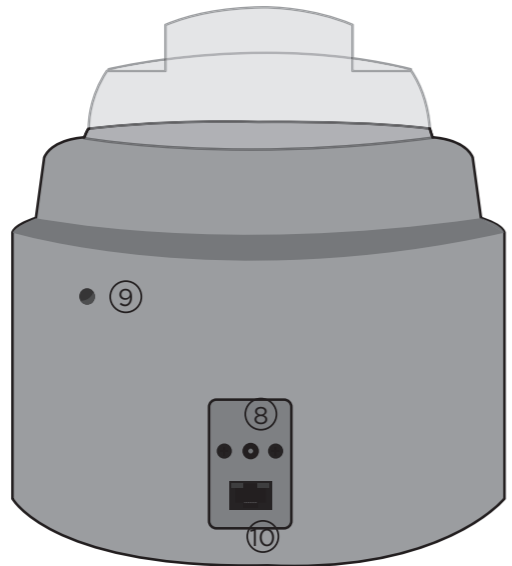
# QUICKSTART GUIDE

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**BIONOVA®**  
IC10/20FRLCD

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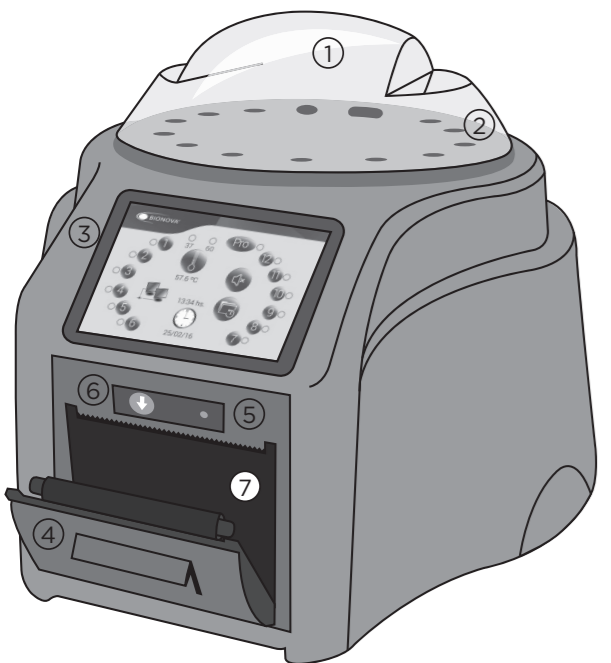
## EN Instructions for use

**Setting Up your Auto-reader**  
Place the Bionova® IC10/20FRLCD Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

**Using your Auto-reader**  
1 | Turn on your IC10/20FRLCD Auto-Reader by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto-Reader. (See image 1)  
4.A | SCBI incubation procedure:  
4.A.1 | To select the incubation program press the button of the corresponding position. Then press the configuration button. A screen will display the different options. Select the SCBI that you are planning to incubate. (See image 3.A)  
4.A.2 | Place the SCBI in the chosen position. The incubation will start automatically. (See image 3.B)  
4.B | ChemDye® PRO1 MICRO Indicator incubation procedure  
4.B.1 | Press the Pro button for 1 second to enter the reading mode. (See image 4.A)  
4.B.2 | Place the protein pen in the corresponding position. Press the @ button to start the incubation. (See image 4.B)  
5 | Wait until the incubation is completed. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. For the PRO1 MICRO indicator, the concentration of the protein detected will also be indicated. If a positive result is detected, press the alarm button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed.  
**Note:** To modify the protein concentration threshold to determine a positive result for the PRO1 MICRO indicator, please refer to the user manual.  
**Disposal:** dispose of the indicators in accordance with your country's waste-disposal guidelines.

**Connectivity**  
To connect your IC10/20FRLCD Auto-reader to a PC via Ethernet, connect an Ethernet cable with a RJ45 connector to the rear of the Auto-reader, then connect the other end of the cable to an Ethernet switch or router. Afterwards, connect your PC to the same Ethernet switch or router using a suitable Ethernet cable. Connecting the Auto-reader to a PC allows the use of the Bionova® Traceability Software, or accessing the Auto-reader's internal web site. Please see the User's Manual for further information.

**Product Compatibility and Incubation programs**  
**Bionova® IC10/20FRLCD Auto-reader Incubation Programs**  
The following incubation programs are available for your Bionova® IC10/20FRLCD Auto-reader.

Temperature	Time	Incubation Program
60 °C	20 min. (Ultra)	20 min. at 60 °C
	30 min.	30 min. at 60 °C
	1 hs	1 hour at 60 °C
	2 hs	2 hours at 60 °C
	3 hs	3 hours at 60 °C
37 °C	PRO (15 min.)	PRO at 60 °C
	4 hs	4 hours at 37 °C

Follow each Indicator's Instructions for use before its incubation.

**Bionova® IC10/20FRLCD Auto-reader compatible indicators**  
We develop new indicators for extending the features of our Auto-readers regularly. Please visit [www.terragine.com/IC1020FRLCD](http://www.terragine.com/IC1020FRLCD) to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible Indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

**Troubleshooting**  
Refer to the user manual for a detailed troubleshooting routine. Please visit [www.terragine.com/IC1020FRLCD](http://www.terragine.com/IC1020FRLCD) or scan the QR code to access the Auto-reader user manual.

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