Bionova® MiniBio Auto-reader

Symbols | Símbolos | Símbolos | Simboli | Symboles

You may see one or more of these symbols on the packaging or labeling of this product. | È possibile que vea uno o más de estos símbolos en el emvase o el etiquetado de este producto. | Poderá ver um ou mais destes símbolos na embalagem ou rotulagem deste produto. | È possibile che uno o più di questi simboli siano presenti sulla confezione o sull'etichetta di questo prodotto. | Vous pouvez voir un ou plusieurs de ces symboles sur l'emballage ou l'étiquetage de ce produit.

	Humidity limitation <p>Limite de humedad Limite de umidade Limiti di umidità Limites d'humidité</p>		For indoor use only <p>Solo se debe usar en interiores Apenas para uso interior Solo per utilizzo all'interno Pour utilisation en intérieur uniquement</p>
	Temperature limit <p>Limite de temperatura Limite de temperatura Limite de temperatura Limite de température</p>		Keep away from sunlight <p>Mantener alejado de la luz solar Evitar exposición a luz Tenere al riparo dalla luce del sole Tenir à l'abri du soleil</p>
	Caution <p>Precaución Precaution Attenzione Attention</p>		Batch code <p>Código de lote Código do lote Numero di lotto Code du lot</p>
	Caution, hot surface <p>Precaución, superficie caliente Attenzione: superficie calda Cuidado, superficie quente Attention , surface chaude</p>		Manufacturer <p>Fabricante Fabricante Producteur Fabricant</p>
	CE Mark <p>Marca CE Marquage CE Marchio CE Marque CE</p>		Contains or presence of natural rubber latex <p>Contiene caucho natural o látex Contém ou presença de látex de borracha natural Contiene o presenza di lattice di gomma naturale Contient ou présente du latex de caoutchouc naturel</p>
	Direct current <p>Corrente continua Corrente diretta Solo corrente continua Courant continu</p>		Recycle electronic equipment <p>Reciclar equipos electrónicos Reciclagem de equipamento eletrônico Smaltimento di apparecchiature elettroniche Recycler les équipements électronique</p>

EN Bionova® MiniBio Auto-reader

Product description
The Bionova® MiniBio Auto-reader was designed for the incubation and automatic readout of Terragene® Rapid, Super Rapid and Ultra Rapid Self-Contained Biological Indicators (SCBIs) for sterilization process control applications.
The Bionova® MiniBio Auto-reader allows two different incubation temperatures, 37 °C or 60 °C. Microorganisms contained within the SCBIs vary depending on the sterilization process for which they have been designed, therefore, incubation temperatures and incubation times differ depending on the SCBI being used. The Bionova® MiniBio Auto-reader allows independent time settings selection for each one of the three incubation positions.
The SCBIs for ethylene oxide (EO), should be incubated at 37 °C, while those used for the monitoring of the rest of the sterilization processes should be incubated at 60 °C.
Note: Simultaneous incubation of SCBIs for monitoring ethylene oxide (EO) processes along with SCBIs for monitoring other sterilization processes is not possible.
The Bionova® MiniBio Auto-reader detects positive and negative SCBIs easily and quickly. A positive result can also be shown by a change in the culture medium color when extended incubations are carried out. Read the Instructions for Use of each SCBI for more information. Due to the high sensitivity of the fluorescence results, the option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital.
The Bionova® MiniBio Auto-reader also provides a printed ticket each time an incubation is completed for recording the results. Allowing easy result management for documentation compliance and record keeping.

Indications for Use
The Terragene® Bionova® Reader Incubators MiniBio incubate at 60 °C and 37 °C and read the Terragene® Bionova® SCBI for fluorescent results at the times prescribed in the User Manuals.

Safety Information
⚠️ To avoid risks and/or damaging the device:
-Use indoors only.
-Do not place the auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
-Do not place the auto-reader near devices that produce strong electromagnetic fields.
-Do not use the auto-reader on sloping surfaces or on surfaces that might be knocked, that may vibrate, be exposed to high temperatures or high relative humidity.
-Disconnect the power cord before cleaning.
-Do not use abrasive, corrosive cleaners or disinfectants.
-Do not immerse into any liquid. Do not pour liquid into it.
-Make sure the auto-reader is connected to an appropriate electrical mains outlet socket.
-Use only the included power supply (AC power adapter), power supply's AC plug, power supply cords, and USB cable. Check that all the items are in good condition every day. If any of them is damaged, stop using them. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injury.
-Do not plug any devices into the auto-reader's USB port other than a personal computer (PC). The PC must be compliant with IEC 60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to check that the device is compatible. Attaching any other device to the USB port may damage the auto-reader and may not be safe for the user.
-Do not attempt to repair the auto-reader yourself. This could lead to major and irreversible damages to the device. If the auto-reader malfunctions, contact your local distributor for further assistance.

⚠️ To reduce the risk of using incompletely sterilized loads:
-Please read, understand, and follow the Instructions for Use of each SCBI before incubating it.
-Do not remove the SCBI before the auto-reader shows the final readout result. Check a result ticket is printed.
-Check that the spore carrier has been completely wetted by culture medium.

⚠️ To avoid the risk of injury by glass fragments produced when crushing the glass ampoule inside the SCBI tube:
-Cool the SCBI during the indicated time before crushing the ampoule.
-Do not handle the SCBI excessively since this might cause the glass ampoule to explode.
-Wear safety gloves and goggles while removing the SCBI from the sterilizer, pressing the SCBI's cap, and crushing the SCBI's ampoule.
-Do not use your finger to crush the SCBI. Use the ampoule crusher instead.

⚠️ To avoid a potentially hazardous situation:
-Avoid contact with the hot metal block inside each incubation position.
-Do not insert your fingers, or any other element, into the incubation positions.
-Place only compatible indicators inside the incubation positions.

⚠️ To avoid SCBIs from absorbing fluorescent particles:
-Avoid direct contact between the SCBIs and chemical indicators or tapes before the SCBI incubation.
-Avoid excessive SCBI handling that may lead to fingerprints or glove talc imprinted over the indicator.
Note: Do not use this product other than in the intended manner specified by Terragene S.A., otherwise the protection provided by the product might be affected.

Note: Authorized personnel may access or service the internal components of the auto-reader. Parts or components inside the auto-reader should not be handled by the user.

Operating conditions <p>Power supply specifications</p>		
Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts
Frequency	50-60	Hertz
Current	1	Amperes
Output parameters	Values	Units
Voltage	12	DC Volts
Current	3	Amperes

Terragene S.A. recommends the use of Uninterruptible Power Supply (UPS) instead of voltage stabilizers, since they fulfill two functions: To stabilize and maintain the energy during a power outage.

Environmental operating conditions		
Environmental conditions	Operating conditions	Units
Altitude	3500 (max.)	Meters
Operation temperature	10-30	Celsius
Relative Humidity	30-80	%
Installation/Over-voltage	Category II	-
Pollution degree	2	-
Storage temperature	5-40	Celsius
Voltage	12	DC Volts

Device safety compliance
The Bionova® MiniBio Auto-reader complies with the following standards and directives:

Electrical Safety
IEC 61010-1
IEC 61010-2-010
Low Voltage Directive 2014/35/EU

Electromagnetic Compatibility (EMC)
EN 61326-1
EMC Directive 2014/30/EU

European Commission
RoHS Directive 2011/65/EU
WEEE Directive 2012/19/EU

The Bionova® MiniBio Auto-reader does not represent photobiological risk and does not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471 standard. Designed under Quality Management System standards ISO 13485:2016/EN ISO 13485:2016.
Note: The auto-reader has been evaluated for conformity for use in business environments. There may be concerns about radio interference if used in home environments. This applies exclusively to the Korean market due to KC certification requirements.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the USB cable from the auto-reader. If the device is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device using a microfiber cloth moistened with a solution of mild dish washing detergent and water. Wring out the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the auto-reader. Then, moisten a clean microfiber cloth with just water, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the auto-reader to air dry for at least 1 hour before connecting the power supply cable or the USB cable again.

This cleaning procedure can be followed whenever considered appropriate. This cleaning procedure must be followed each time a spill occurs on any of the external surfaces of the auto-reader. If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.
Note: Do not clean the internal parts of the device.
Note: Do not pour liquid onto the device or immerse it in any liquid. Do not allow any liquid to enter the device while it is being cleaned.

Maintenance
The auto-reader does not require routine maintenance.

Warranty
The products are guaranteed to be free from materials and workmanship defects when properly installed, maintained and used for their intended purpose indicated in the applicable product label and/or the User's Guide. The warranty only applies to the original purchaser.

Term
The warranty period for the Bionova® MiniBio Auto-reader is 1 (one) year from the date of installation, and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability
Terragene S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
For information regarding the functioning or condition of the product, contact Terragene S.A. directly to info@terragene.com or through our website www.terragene.com. In USA, please call +1 844 837 7243.
Terragene S.A.
Ruta Nacional No 9, Km 280 - CP 2130
Parque Industrial Micropi-Alvear-Santa Fe-Argentina.

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EN Instructions for use

Note: Remember to set-up the auto-reader local date and time before starting incubations.

Setting up your auto-reader

Place the auto-reader on a firm surface, free from vibrations, away from direct sunlight, hot or cold air flows, chemical and corrosive or flammable substances. Do not place the auto-reader in a way that would make it difficult to remove the supply's AC plug from mains. Leave a space of at least 10 cm from the auto-reader to the closest wall. Do not move the auto-reader periodically or while it is being used. Connect the auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning it on, check that all reading positions are empty.

Using your auto-reader

- 1 Switch on your auto-reader by connecting the AC plug of the supply's AC plug to the mains and then connect the plug at the other end of power supply to the rear of the auto-reader. (See image 1)
- 2 Press the incubation temperature selection button for 3 seconds to enter the temperature selection mode. Press again to select the desired temperature. After 4 seconds the temperature modification will be accepted. (See image 2)
- 3 Wait until the temperature stability indicator light stops blinking. Select the appropriate incubation time for the SCBI you are planning to incubate by pressing the incubation time selection button. (See image 3)
- 4 Place an SCBI in the chosen position. The incubation process will start automatically. (See image 4)
- 5 Wait until the incubation has finished. The negative indicator light will turn green to indicate a negative result and the positive indicator light will turn red to indicate a positive result. If a positive result is detected, press the alarm cancellation button to cancel the alarm. Do not remove the SCBI until the ticket with the result of the incubation is printed.

Disposal: Dispose of the SCBIs in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your auto-reader to a personal computer (PC) by USB, connect one end of the USB cable provided to the rear part of the auto-reader and the other end to the USB port on your PC. Connect the auto-reader to a PC to use the Bionova® Cloud Environment. Please refer to the User Manual for more information.

Product compatibility and incubation programs

The following incubation programs are available for your auto-reader.

Temperature selection	Time selection	Incubation program
60 °C	20 minutes 30 minutes	20 minutos a 60 °C 30 minutos a 60 °C
	1 hour 2 hours 3 hours	1 hour at 60 °C 2 hours at 60 °C 3 hours at 60 °C
37 °C	4 hours	4 hours at 37 °C

Note: Please read and follow the SCBI Instructions for Use before using it.
Note: Select the appropriate incubation program before the incubation of any indicator.

Troubleshooting

Refer to the User Manual for detailed troubleshooting measures.
Please visit www.terragene.com/MINIBIO or scan the QR code to access the auto-reader User Manual.

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