
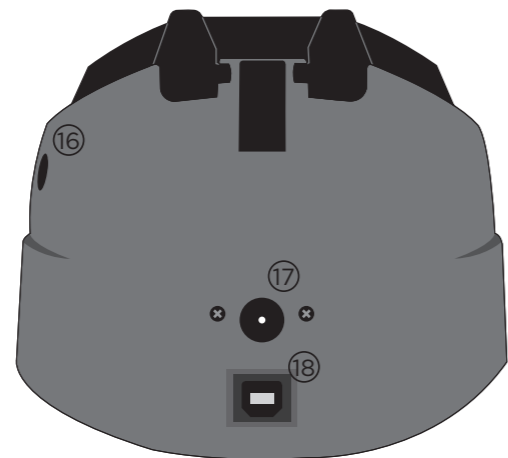
QUICKSTART GUIDE

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BIONOVA®
MiniPro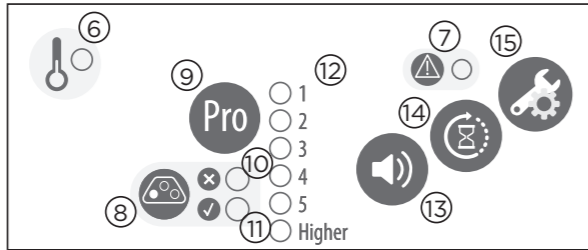

Terragene®
Rev. 2 | December 2021

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EN Instructions for use

Setting Up your Auto-reader

Place the Bionova® MiniPro Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto-reader

1) Turn on your MiniPro Auto-Reader by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto-Reader. (See image 1)
2) Wait until the temperature stability indicator stops blinking. Press the Pro button of the selected position for 1 second to enter the reading mode. (See image 2)
3) Wait until the qualitative indicator of protein concentration corresponding to 1 µg stops blinking. Place the protein pen in the corresponding position. Press the Pro button again for 2 seconds to start the incubation. (See image 3)
4) Wait until the incubation is completed. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. For the PRO1 MICRO indicator, the concentration of the protein detected will also be indicated. If a positive result is detected, press the alarm button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed.
Note: To modify the protein concentration threshold to determine a positive result for the PRO1 MICRO Indicator, please see the user's manual.
Disposal: Dispose of the indicators in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your MiniPro Auto-reader to a PC via a USB port, connect an end of the USB cable (included) to the rear part of the incubator and the other end to the USB port of your PC. Connecting the Auto-reader to a PC allows the use of the Bionova® Traceability. Please see the User's Manual for further information.

Product Compatibility and Incubation programs

Bionova® MiniPro Auto-reader Incubation Programs

The following incubation programs are available for your Bionova® MiniPro Auto-reader.

Temperature	Time	Incubation Program
60 °C	PRO (10 min.)	PRO at 60 °C

Follow each Indicator's Instructions for use before its incubation.

Bionova® MiniPro Auto-reader compatible indicators

We develop new indicators for extending the features of our Auto-readers regularly. Please visit www.terragene.com/MINIPRO to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

Troubleshooting

Refer to the user manual for a detailed troubleshooting routine. Please visit www.terragene.com/MINIPRO or scan the QR code to access the Auto-reader user manual.

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