Bionova® Photon Instant Auto-reader

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EN Bionova® Photon Instant Auto-reader

Product description

Bionova® Photon Instant Auto-reader has been designed for the incubation and automatic readout of the Bionova® Photon line of Self-contained Biological Indicators (SCBIs) appropriate for monitoring Steam Sterilization Cycles.

Photon Instant Auto-reader presents two independent readout positions. Each position automatically detects whenever a SCBI is placed for incubation and delivers a readout result after a few seconds using advanced fluorescence techniques.

Bionova® Photon Instant Auto-reader allows easy and rapid detection of positive and negative SCBIs. A positive result, can also be evidenced by culture medium color change when performing extended incubations. Read the instructions for use of each SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital.

Photon Instant Auto-reader has built-in USB, Wi-Fi, and Bluetooth® technology connectivity capabilities. The electronic tickets (e-tickets) created for each readout process can be accessed using a compatible device thus allowing easy documentation compliance and traceability.

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EN Bionova® Photon Instant Auto-reader

Product description
Bionova® Photon Instant Auto-reader has been designed for the incubation and automatic readout of the Bionova® Photon line of Self-contained Biological Indicators (SCBIs) appropriate for monitoring Steam Sterilization Cycles.

Photon Instant Auto-reader presents two independent readout positions. Each position automatically detects whenever a SCBI is placed for incubation and delivers a readout result after a few seconds using advanced fluorescence techniques.

Bionova® Photon Instant Auto-reader allows easy and rapid detection of positive and negative SCBIs. A positive result, can also be evidenced by culture medium color change when performing extended incubations. Read the instructions for use of each SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital.

Photon Instant Auto-reader has built-in USB, Wi-Fi, and Bluetooth® technology connectivity capabilities. The electronic tickets (e-tickets) created for each readout process can be accessed using a compatible device thus allowing easy documentation compliance and traceability.

Indications for Use
Terragene® Bionova® Photon Instant Auto-reader incubates at 60 °C and reads the Terragene® Bionova® Photon SCBIs at the times prescribed in the User Manual.

Safety Information
To avoid risks and/or damaging the device:

- For indoor use only.
- Do not place the Auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
- Do not place the Auto-reader near devices that emit strong electromagnetic fields.
- Do not use the Auto-reader on leanned surfaces or on surfaces that are subjected to shocks, vibrations, temperature or high relative humidity.
- Disconnect the power cord before cleaning.
- Do not use abrasive or corrosive cleaners or disinfectants.
- Do not immerse into any liquid. Do not pour any liquid inside.
- Make sure the Auto-reader is connected to an appropriate electrical mains outlet socket.
- Use only the included power supply (AC power adapter), power supply's AC plug, power supply cords, and USB cable. Verify that all the included elements are in good condition on daily basis. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.
- Do not attempt to repair the Auto-reader by yourself. That could lead to major and irreversible damages to the device. In case of device malfunction, contact your local distributor for further assistance.
- Do not plug any devices into the Incubator's USB port other than a personal computer (PC). The PC has to be compliant with IEC60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its USB ports. Ask a qualified technician to verify device compatibility. Attaching any other device to the USB port may damage the incubator and may not be safe for the user.

Technical assistance
Terragene® S.A. Ruta Nacional N° 9, Km 280 - CP 2130. Parque Industrial Micropi-Alvear-Santa Fe-Argentina.

Warranty
Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period, the only obligation of Terragene® S.A. will be product repairing or substitution.

Term
The warranty period for electronic products marketed by Terragene® will be 1 (one) year from the date of first use of the product, and may never be extended beyond the product's useful life -5 (five) years from its manufacture-.

Limitation of liability
Terragene® S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

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Operating conditions
Power supply specifications

Input parameters	Operating Conditions	Units
Voltage range	100-240)	AC Volts
Frequency	50/60	Hertz
Current	0,6	Amperes
Output parameters	Values	Units
Voltage range	12	DC Volts
Current	1,5	Amperes

Terragene® S.A. recommends the use of UPS instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

Environmental operating conditions

Environmental operating conditions	Operating Conditions	Units
Altitude	3500 (máx.)	Meters
Operating temperature	10-30	Celsius
Relative humidity	30-80	%
Installation/over-voltage	Category II	
Degree of contamination	2	
Storage temperature	10-30	Celsius
Voltage	12	DC Volts

Regulatory compliance
EU Wireless Compliance: Hereby, Terragene® S.A. declares that the radio equipment type Bionova® Photon Instant Auto-reader complies with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: www.terragene.com/DOC.

RF transmitter specifications: Wi-Fi transmitter total output power 19,9 dBm E.I.R.P (97,72 mW) in the frequency band of 2,4 GHz ISM band / 2,412 to 2,462 GHz (Channels 1 to 11). Bluetooth® transmitter total output power 4,6 dBm E.I.R.P (2,88 mW) in the frequency band of 2,4 GHz ISM band / 2,402 to 2,480 GHz.

FCC/IC Wireless Compliance: Hereby, Terragene® S.A. declares that the radio equipment type Bionova® Photon Instant Auto-reader complies with applicable FCC/IC directives. Contains certified transmitter module: FCC ID: 2AC7Z-ESPWROOM32D / IC ID: 21098- ESPWROOM32D.

This device complies with Part 15 of the FCC Rules and Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016. For more information about the device wireless and safety regulatory compliance, please refer to the Auto-reader User Manual.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the USB cable from the Auto-reader. If the device is hot, wait until it has cooled down before handling it. Clean the external surfaces of the device using a microfibr cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the Auto-reader. Afterward, moisten a clean microfibr cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Auto-reader to air dry for at least 1 hour before connecting the power supply cable or the USB cable again. This cleaning procedure can be followed whenever considered appropriate. This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the Auto-reader. If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

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EN Instructions for use

Setting Up your Auto-reader

Place the Bionova® Photon Instant Auto-reader on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or during its use. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto-reader

- 1] Power on your Photon Instant Auto-reader by connecting the power supply's AC plug to mains and then connect the plug at the other end of power supply to the rear of the Auto-reader. (See image 1)
- 2] Wait until the Temperature Progress & Stability Indicator stops blinking. The Instant logo will turn on when the Auto-reader is ready to process a sample. Place a SCBI in a readout position. The incubation will start automatically. (See image 2)
- 3] Wait until the incubation is complete. The position status indicator will turn green to indicate a negative result, or red to indicate a positive result. If a positive result is found, press the alarm/configuration button to silence the alarm. Do not remove the SCBI until a result is informed. (See image 3)
- 4] Run Bionova® Traceability Software or the Bionova® Wireless Assistant mobile app to check the incubation result e-ticket.

Disposal: Discard the SCBIs following your Country disposal directives.

Connectivity

To connect your Photon Instant Auto-reader to a PC by USB, connect one end of the included USB cable to the rear of the incubator and the other end to an USB port on your PC. Wi-Fi connectivity is also available and can be Set-up using the Bionova® Wireless Assistant. Connecting the Auto-reader to a PC (by USB, or Wi-Fi) allows using Bionova® Traceability Software to save, manage, and print e-tickets reports. Connecting the Auto-reader to an Android device (by Wi-Fi or Bluetooth®) allows using the App to check the status of the device, access e-tickets, and more. Check the User Manual for more information.

Product Compatibility and Incubation programs

Bionova® Photon Instant Auto-reader Incubation Programs

The following incubation programs are available for your Bionova® Photon Instant Auto-reader.

Temperature	Time	Incubation Program
60 °C	Instant (7 sec)	Instant at 60 °C

! Follow each Indicator's instructions for use before its incubation.

Bionova® Photon Instant Auto-reader compatible indicators

We develop new indicators for extending the features of our Auto-readers regularly. Please visit www.terragene.com/INSTANT to access the Compatibility Matrix for Terragene® Auto-readers. The Compatibility Matrix specifies all compatible Indicators developed for your Auto-reader up to date. Stay tuned for new and exciting updates!

Troubleshooting

Refer to the user manual for a detailed troubleshooting routine. Please visit www.terragene.com/INSTANT or scan the QR code to access the Auto-reader user manual.

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