Automatic System

for Quality Control and Traceability of Cleaning,
Disinfection and Sterilization Processes.


Innovation in technologies for Infection Control

TERRAGENE®

Rev.1 | December 2019

Composition	4
References	4
Description of use	4
Safety Information	5
Symbols	5
Power supply specifications	6
Environmental operating conditions	6
Series/Lot number	6
System requirements	6
Instructions for Use	7
Start-up	7
Interpretation of Results	8
Indicators Disposal	8
Care & Cleaning	8
Troubleshooting Table	10
Warranty	11
Liability Limitations	11
Technical Assistance	11

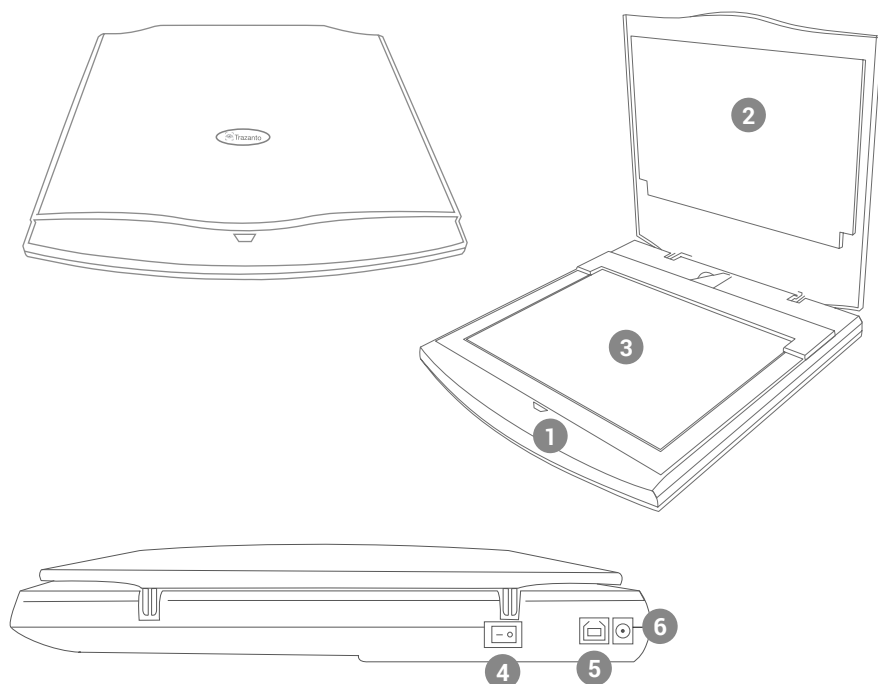References

- 1 Power LED indicator
- 2 Cover
- 3 Glass
- 4 Power button
- 5 USB port
- 6 Power supply input

Description of use

Trazanto Automatic System for Quality Control and Traceability of Cleaning, Disinfection, and Sterilization Processes has been designed to assist the user in the processing of Terragene® indicators in a quick and easy manner.

To access the complete list of indicators compatible with the system, please visit www.terragene.com.ar/trazanto

Trazanto is a system comprised of two fundamental modules: a software-implemented Analysis module, and a hardware module.

The hardware module is composed of the Trazanto Optical Reader, an optical image acquisition device carefully controlled and calibrated to achieve reliability and repeatability of all measurements performed on compatible indicators.

The Analysis Module has been designed to work together with the Trazanto reader. The module performs the readout and detection of compatible indicators through the use of advanced intelligent algorithms, which retrieve and digitize all the relevant indicator information and display it conveniently, facilitating the identification of approved and rejected indicators.

Trazanto speeds up the process of registering the indicators for future use by allowing the data digitized by the analysis module to be registered for traceability using appropriate software. Trazanto is fully compatible with Bionova® Traceability Software. Trazanto's Analysis Module is distributed with the latest version of Bionova® Traceability Software. Follow the instructions in the *Start-up* section to download and install the software.

Simultaneous readout of more than one indicator is not possible.

Symbols

Operating relative humidity.

Operating environmental temperature.

Caution, warning.

Important, attention.

Direct current.

For indoor use.

Keep away from sunlight.

Manufacturer.

Serial number.

Lot number.

Separate collection for waste of electrical and electronic equipment

Trazanto's LED light sources do not represent photobiological risk and do not generate dangerous optical radiation in any of their operational conditions.

To avoid risks and damaging the equipment

- For indoor use only
- Do not hit the glass, it is fragile and may break.

To avoid automatic readout errors

- Verify that moisture condensation is not present on the reader's glass.
- Verify that the glass is free of dirt, stains, dust, etc.
- Do not open the cover during a readout.

- Unplug the device if you are not going to use it for a long time (during the night or the weekend) to prevent the risk of fire.
- Do not place the device in a room exposed to direct sunlight or to high luminous intensity lamps.
- Do not place the reader near devices that emit strong magnetic fields.
- Do not use the device on leaned surfaces or on surfaces that are subject to shocks, vibrations, temperature or high relative humidity.
- Do not use the device near to places with dust water or heat.
- Do not pour any liquid inside.
- Do not submerge the device into any liquid.
- Make sure that the device is connected to the correct power supply.
- Only use the AC power adapter included with the device. The use of AC power adapters different from the one given may cause fires, electrical shocks, or even physical injuries.
- Only use the AC power cord and the USB cable included with the device. Verify that the cables are in good condition on a daily basis. If the cables are damaged, discontinue their use. The use of AC power cords and/or USB cables different from the ones given may cause fires, electrical shocks, or even physical injuries.
- Unplug the power supply cord before cleaning the device.
- Do not plug any devices into the Trazanto reader USB port other than a personal computer. The computer has to be compliant with IEC 60950-1, IEC 62368-1, or comparable with safety extra-low voltages on the USB ports. Ask a qualified technician to verify equipment compatibility. Attaching any other device to the USB port may damage the reader and may not be safe for the user.
- Do not use abrasive, corrosive cleaners or disinfectants.
- In case of a technical failure, contact the manufacturer for advice. Please, do not open and/or try to fix the device on your own, this will void the warranty and may cause a bigger and irreversible damage.

- Verify that after the cleaning or sterilization process the indicator does not show alterations such as scratches, tears, other mechanical failures and/or alterations in its plastic laminate.
- Do not write on the indicators before performing the readout.

Do not use this product in a way different from the one specified by Terragene® S.A. Otherwise, the protection given may be affected. Only personnel authorized by Terragene® S.A. should open the equipment and have access to its components. No repairs should be made by the user.

Power supply specifications

Input parameters	Operating Conditions	Units
Voltage range	100-240	ACVolt
Frequency	50/60	Hertz
Current	0.6	Amperes
Output parameters	Operating Conditions	Units
Voltage	24	DC Volts
Current	0.75	Amperes

Terragene® recommends the use of UPS instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

Environmental operating conditions

Environmental conditions	Operating Conditions	Units
Altitude	3500 (máx.)	Meters
Operating temperature	10-30	Celcius
Relative humidity	30-80	%
Voltage installation	Category II	
Storage temperature	10-30	Celcius
Voltage	24	DC Volts

Series/Lot number

For an easy identification, every Trazanto reader has a unique series number printed on a label located on the bottom of the device. Please, record your series number on this guide for future reference:

Serial number: _____

Batch number: _____

System requirements

These are the minimum hardware requirements, as a general guidance:

CPU: Dual core, 2GHz

RAM Memory: 32 bit: 2 GB / 64 bit: 4 GB

USB Port: USB 2.0 (compatible with USB 3.0)

Operative system: Windows 7 (SP1) / 8 / 10

Start-up


1- Place the Trazanto reader on a firm surface, without vibrations, away from sunlight and from chemical products that are either abrasive or flammable. Place the device with the cover facing upwards, and the power LED indicator pointing towards you. Place the device in a way that it is not difficult to unplug it from the power supply input. Leave at least 10 cm between the device and the wall. Do not move the device on a regular basis or during its use. Connect the device to a safe and stable electric power source.

Do not wet or heat the device. If any liquid is spilled over the device, unplug it and dry it immediately.

2- In order to use Trazanto together with Bionova® Traceability Software, download the latest available version of the traceability software from the following link <http://www.terragen.com.ar/software>

Once the file has been downloaded, run the installer and follow the instructions on the screen, verifying that the option "Install Trazanto" is selected at the time of installation.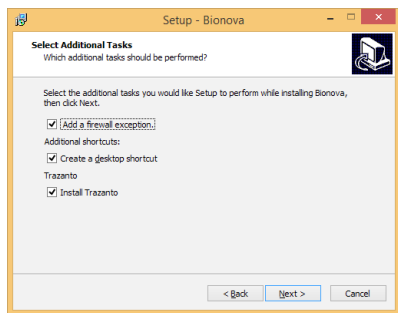


NOTE: To make sure that Trazanto reader will be detected properly, install the software before connecting it to the PC for the first time.

3- Turn on the device by connecting the AC power supply to mains, and the other end (plug) on the back of the device. Press the power button on the back of the device to the position "I". The power LED indicator will turn on indicating that the device is ready for use.

NOTE: To turn off the device, press again the power button to the position "O".

4- Connect the square end of the USB cable to the USB port of the device, and the rectangular end to the USB port of the PC.

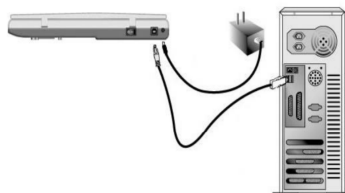5- Run the Bionova® Traceability Software.

6- Open the cover of the device. Put a chemical or cleaning indicator on the glass with the ink facing downwards. It is not necessary to line up the indicator.

IMPORTANT:


Put only one indicator every time an automatic readout is performed.

For a correct readout, the glass must be free of condensation, dirt, stains, dust, digital prints, etc. If the glass is not clean, please follow the cleaning procedures described in *Care & cleaning* section

Do not write on the indicator before performing its automatic readout. Any written marks on the indicator may invalidate the readout process.

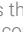Start the automatic readout process. For doing so, close the cover of the device and, once in the Bionova® Traceability Software, go to the TRAZANTO tab or press the icon  on the task bar. Press the button ANALYZE to star the readout. If it runs correctly, a window as the one below will pop up, with all the information of the analyzed indicator and the readout results.

⚠️ Verify that after the sterilization process the indicator does not have scratches, tears, or air bubbles on the ink. Given that this could influence the automatic readout process, check the results visually.

Interpretation of Results

Automatic Readout

Read carefully the instructions for use of every indicator before performing its automatic readout. In all cases check the results visually. The results of an indicator's readout may be:

Rejected The reader detects that the indicator has not completely changed color.

Accepted The reader detects that the indicator has completely changed color.

The automatic readout process requires an appropriate preparation of the indicator. Every time an indicator is read, verify that the image result is clear enough, and that it does not have stains external to the indicator. If you see that the image result is not clear enough, has stains or visible dirt, please follow the cleaning procedures described in *Care & Cleaning* section. Afterwards, perform again the automatic readout of the indicator.

Visual readout

Use the result's reference guide that is included in every indicator's instruction for use when verifying the results visually

Indicators Disposal

Discard the indicators after its use, according to the sanitary regulations of your country. It is not possible to reuse the indicators.

Care & Cleaning

Before performing any cleaning procedures on the Trazanto Reader, turn off the device and unplug the power and USB cables from the back panel. Wait a few minutes until the device is at room temperature before cleaning it.

Cleaning of external surfaces

Clean the external surfaces with a slightly damp cloth. Do not allow any liquid to run inside the device. Do not clean the inside of the device. If the internal components require a general cleaning, please contact your local distributor.

Depending on the type of indicator, the following fields will be automatically filled in:

Manufacturer, date of expiration, date of manufacture, brand, product code and lot. The result will be informed over the image of the analyzed indicator. Additionally, complete the required process fields.

Check the result of the indicator visually. If your visual result matches that of the automatic readout, press the *Agree* button to save the result. If your visual result does not match that of the automatic readout, press the *Disagree* button to save the result of your visual exam.

To discard the automatic readout of an indicator, press the button

- ⚠️ The first time the reader is used to perform the analysis of an indicator, Trazanto will connect to the Internet once only to download the configuration files necessary for its operation. Verify that your PC has an active Internet connection.
- ⚠️ Read carefully the instructions for use of the indicators you use. Use the results' reference guide that is included in every indicator's instruction for use when you have to verify the result visually.

Cleaning of the glass

The glass should be clean and free from stains in order to ensure the quality of the indicator's analysis.

Procedure:

1. Turn off the Trazanto Reader, and unplug the power and USB cables from the device.
2. Wait until the glass is at room temperature.
3. Slightly moisten the included microfiber cloth with isopropyl alcohol (95%). Verify that the cloth is clean. Do NOT use water.
4. Open the cover and carefully clean the glass. Use gloves.
5. Wait a few minutes while the alcohol evaporates. Verify that there are no stains or remains left on the glass.
6. Close the cover.

Do not clean the glass if the device is at a temperature higher than room temperature, since this could cause serious damages to the device.

Troubleshooting Table

Failure	Possible cause	Action
Trazanto reader does not turn on.	The power supply is not properly connected to the reader.	<p>Verify that the power supply is connected to a proper mains supply.</p> <p>Check that the power supply plug is properly connected to the reader.</p>
Can't access to Trazanto from Bionova® Traceability Software.	Problems during the installation of Trazanto system.	Download and install the latest version of Bionova® Traceability Software. Verify that the check box "Install Trazanto" is checked at the time of installation.
Can't establish connection with the reader. <i>Error message "Reader not detected"</i>	<p>USB cable USB not connected / not properly connected</p> <p>Problems during the installation of Trazanto system.</p>	<p>Check the USB cable connection. Try to unplug all the USB cable connectors and plug them in again.</p> <p>Verify that Trazanto system has been properly installed.</p> <p>Verify that the reader has been properly recognized and installed by the operating system.</p>
There is a black line through the digitalized image.	Communication problem with the reader.	Reset the reader. Press the power button to position "0", wait 10 seconds, and press again the button to position "1".
The digitalized image has marks that are not in the indicator.	There is dirt on the glass.	Clean the glass as indicated in the <i>Care & Cleaning</i> section.
Trazanto cannot analyze an indicator for the first time. <i>Message "Calibration files have not been found"</i>	<p>There is dirt on the glass.</p> <p>The reader's configuration files have not been downloaded.</p>	Verify that your PC has internet access. Close the traceability software, and follow the steps in the <i>Start-up</i> section
Trazanto cannot analyze properly an indicator. <i>Message "Verify that the indicator is compatible with the system"</i>	<p>There is no indicator on the reader.</p> <p>Partial readout of an indicator/ Indicator incorrectly placed.</p> <p>Analysis of a non-compatible indicator.</p> <p>Analysis of a compatible indicator but it is damaged or altered.</p>	<p>Verify that the indicator has been correctly placed.</p> <p>Verify that the glass is clean. Verify that the indicator is correctly placed on the glass.</p> <p>Verify that the indicator used says "Trazanto". Check the list of indicators compatible with the system.</p> <p>Verify that the indicator is not altered (it should not have scratches, tears, air bubbles, etc.) and that it is not written. If the indicator is altered, check the results visually.</p>

Warranty

Terragene® S.A. guarantees the quality of both the product's components and their manufacturing process. Should the product present any material or manufacturing defects within the warranty period (one year from its purchase) Terragene®'s only obligation will be its repair or replacement.

Liability Limitations

Terragene® S.A. is not liable for possible losses or damages resulting from the use of this device, from the user's negligence or his strict responsibility.

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