UVFlow

Operating, Maintenance & User Guide

Rev. 0 | August 2021

Copyrights

This operating, maintenance and user guide contains confidential information that is protected by copyrights.

None of the parts of this operating, maintenance and user guide may be photocopied or copied without the prior written notice by Terragene®.

The information contained in this document has only informational purposes and is subjected to changes without previous notice. Terragene® takes no responsibility for errors or inaccuracies that might appear on this guide.

Download the latest version of the user's manual and other resources for your equipment here.

terrogene.com/download

Index

4 **Product description**

- 4 Introduction
- 4 General description

5 **Safety Information**

- 5 Terminology and safety symbols
- 5 Safety
- 6 Precautions to be taken before working
- 6 Precautions during work
- 6 Recycling Guidelines

6 **Installation**

- 6 Electrical Installation
- 6 Electrical Connections

6 **Use**

- 7 On, Off, and Automatic mode
- 7 UVFlow with push-button or manual mode
- 7 UVFlow with remote control via connection to your internal network or remote mode
- 8 Connecting the UVFlow to your local Wi-Fi internet network

8 **Maintenance and repair**

- 9 Instructions for changing the germicidal tube

9 **Transportation and storage**

- 9 Delivery inspection
- 9 Unit Inspection
- 9 Transport Guidelines
- 9 Storage Guidelines

10 **Product warranty**

- 10 Liability
- 10 Warranty Claims
- 10 Coverage and Limitations

■ Product description

Introduction

This operations, maintenance and usage manual is designed to help you achieve the best performance and long life for your UVFlow.

The UVFlow disinfection system is designed to be used as part of a cleaning and disinfection program to eliminate pathogens that may be present in the air of the room where it is placed. The UVFlow emits UV-C energy of 253.7 nanometers wavelength, which makes it highly germicidal. The optimal quality of the germicidal tubes prevents the emission of ozone or other secondary pollutants.

In addition, people can be in the room while the equipment is in use since the energy is confined inside the equipment, not releasing any form of radiation to the outside (*tested by UVC LINYANG LS125-UVCWP-X1-PROBE UVC radiometer equipment*). The UVFlow has the additional advantage of filtering and treating the air, emphasizing that, in this way, it can be used with people inside the room.

Our UVFlow disinfection system is a equipment designed for fixed and wall mounting. It has a modern design and is fully functional, and the unit is sturdy enough for daily use.


Its main advantages are:

- 99.99 % disinfection through the use of confined germicidal lamps.
- Air free of all pathogens.
- Renewal and cleaning of the air in the room.
- Greater safety for people and their health.
- Better hygiene of facilities in general.

If you have any questions concerning this equipment or its application not covered in this manual, please contact your local Terragene® distributor.

General description

Our Terragene® UVFlow disinfection system is a fixed electronic equipment consisting of a wall-mounted fixture. It consists of air blowers and a manual controller. The UVFlow system is designed to be used as part of a continuous disinfection program. It removes pathogens from an enclosed environment by exhausting air and passing through UVC germicidal lights.

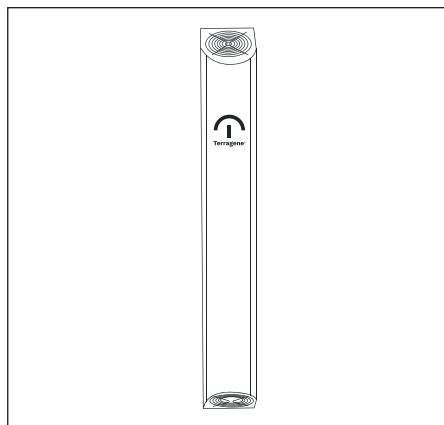Casing

- UVC radiation-resistant casing.
- Natural terminations.

Fan

- Cooling fan of the internal elements.
- Low noise level.
- Low vibration level.

UVC lamps

- Germicidal frequency lamps of 254 nanometers.
- 30 watts lamps.
- Positioned to optimize disinfection at 360°.
- 9.000 hours of use.
- No ozone production / insignificant ozone production level.

Precautions to take into account:

- Do NOT place objects or living beings below or above the fans to allow the airflow to run fluently towards the equipment.
- Airflow should run freely.

Product description

- DO NOT place the equipment on chairs and/or places where living beings could circulate.
- DO NOT look directly over the fans.

The UVFlow is an electrical unit measured by UVC radiometer LINYANG LS125-UVCWP-X1-PROBE which ensures that this unit is safe and fulfills its purpose of protecting people.

Safety Information

Terminology and safety symbols

About safety messages: It is essential that you carefully read, understand, and follow the safety messages and regulations before handling the product. These are published to prevent these risks:

- Personal accidents and health problems.
- Damage to the product.
- Product malfunction.

Levels of danger

Danger: A hazardous situation that, if not avoided, will result in serious injury.

Warning: A hazardous situation that, if not avoided, may result in injury.

Caution: A hazardous situation which, if not avoided, may result in minor or moderate injury.

Caution:

- A potential situation, which, if not avoided, could lead to undesirable results or conditions.
- A practice that is not related to personal injury.

Safety

Caution: Read this entire manual carefully before using the

equipment. It contains information on how to use the product and component changes when changes are necessary.

The following types of messages are used in this manual to alert maintenance personnel to procedures that require special attention for the protection and safety of both personnel and equipment:

Caution: Read this manual carefully before installing and using the product. Improper use of this product can result in personal injury and property damage and may void the warranty.

Save this manual for future reference and keep it handy where the unit is located.

Warning:

- The operator must consider the safety precautions to avoid physical injury.
- This equipment is not suitable for use by people with physical or mental disabilities or lack of experience unless they are under the supervision of a responsible person.
- Children should not play with the equipment.
- In the event of a damaged power supply cord, it must be replaced immediately by trained personnel or technical service.
- This equipment has light tubes containing mercury gases. In case of breakage, dispose of them and ventilate the environment immediately.
- Operation or maintenance of the unit in any manner not specified in this manual may result in damage to the equipment, serious injury, or death. It also includes any modification to the equipment or use of parts not supplied by Terragene®. If you have any questions regarding the intended use of the equipment, please contact a Terragene® representative before proceeding.
- This manual identifies the accepted methods for disassembling the units. These methods must be followed. Never apply any force on the equipment to facilitate removal, unless explicitly specified in this manual.
- If any part of the equipment is damaged, do not use it, as this may result in electric shock, fire, explosion, release of toxic gases, physical injury, or damage to the environment. Correct or repair the problem before operating the equipment.
- Do not change the service application without the approval of an authorized Terragene® representative.

All Terragene® products are designed with the highest quality standards to perform on their own in such sensitive spaces as the home, commercial, and healthcare spaces.

All Terragene® equipments have timers and/or remote ignition and do not need to be manipulated while they are on.

⚡ Safety Information

Despite the above, it is necessary to follow the user manual of each product. The safety of our users and customers is our top priority.

Hazard Categories

Hazard categories may be under hazard levels or allow specific symbols to replace common hazard level symbols.

Electrical hazards are indicated by the following specific symbol:

Electrical Safety

Please note these safety regulations regarding waste and emissions:

- Dispose of all waste properly.
- Handle and dispose of the electrical waste following applicable environmental regulations.

Precautions to be taken before working

Please note these safety precautions before working with the product or when interacting with the product:

- Check that the equipment has a charge in its battery and/or is properly connected to the electrical power source.

Precautions during work

Please note these safety precautions when working with or coming into contact with the product:

Caution: Read this manual carefully before installing and using the product. Improper use of this product may result in personal injury and property damage and may void the warranty.

MANUAL UNIT: UVFLOW

This unit is set to work in "on" mode while there are people inside the room where it is placed, as the UVC lamps are enclosed inside the unit.

Under no circumstances operate the UVC lamps without the casing or cover of the unit.

It is necessary to warn about this situation.

Irresponsible exposure to c-type UVC lamps, which are part of Terragene® products, may have some effects such as:

- Indiscriminate exposure to the rays can increase the risk of skin conditions in the same way that sunlight can.
- If you use ultraviolet light on the skin in a systematic way for extended periods, it can increase the risk of skin cancer.
- Can cause premature skin aging and signs of sunlight damage such as freckles, skin thickening, actinic keratosis or solar elastosis.
- May cause corneal inflammation or burning with prolonged exposure.
- May contribute to cataracts and tissue growth on the surface of the eye.

Under no circumstances should ultraviolet light be handled while it is on, unless certified UVC protection equipments are available. It is mandatory to follow the instructions in the user's manual to ensure complete user safety.

Recycling Guidelines

Always follow local rules and regulations regarding recycling.

■ Installation

Electrical Installation

For electrical installation recycling requirements, consult your local power company.

Electrical Connections

Electrical connections must be made by licensed electricians following all local, state, national and international regulations. For more information on requirements, refer to the sections relating specifically to electrical connections.

■ Use

The UVFlow can be used in the presence of people.

It is suitable for use in any space where you want to protect people's health.

It has two ways to use it, by manual ignition and by ignition by mobile equipment.

Both uses are detailed below:

On, Off, and Automatic mode

The instructions herein may vary depending on the version of UVFlow you have purchased.

We recommend that you read the operating instructions that came with the product so that you can use the most current version of your product.

Please send a message to info@terragene.com to find out if a more current version of the operating software is available.

The UVFlow has two different ways to turn it on:

- By pushing a button or manual mode.
- By connecting to the equipment's network or remote mode.

UVFlow with push-button or manual mode

On the bottom of the equipment, you will find a push-button that enables the unit's on, off, and intermittent mode functionalities.

Push-button operating modes:


- 1|Single push (1 click): Continuous On mode | BLUE LED.**
In this mode, the equipment will remain on, sanitizing air continuously, and the LED of the equipment will be on in blue color.
- 2|Double click (2 quick clicks): Continuous intermittent cycling mode (30 min. ON / 15 min. OFF) | VIOLET OR GREEN LED depending on the equipment.**
The automatic mode programmed in the equipment is 30 minutes of disinfection for every 15 minutes off. This programming works as long as the equipment is connected and has power. This programming was set according to the optimal duration of time that a place should have its air recirculated and sanitized.
- 3|Long push (more than 2 seconds): Off Mode | RED LED.**

UVFlow with remote control via connection to your internal network or remote mode

The remote mode allows you to use the MODE: ON, OFF, and INTERMITTENT CYCLE through a mobile equipment like a cell phone, tablet, pc, etc.

Steps to follow to achieve the connection:


- 1|Identify the network of the equipment. It will look as follows.**

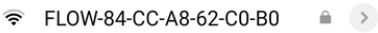(This is an example)

- 2|Connect to the Wi-Fi network generated by the equipment, using the credentials (username and password) listed on this quick connect sheet or on the box on your computer.**

FLOW-84-CC-A8-62-C2-74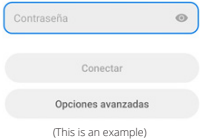

(This is an example)


Password:
terragene

- Note:** You can use the QR code to connect directly.
- 3|Once the connection is established, verify that your mobile equipment is still connected to this network (since this Wi-Fi network is only for internal connectivity and does not provide internet access).** When connecting, the equipment will redirect you to the internal network that controls the equipment's ignition. Each time you connect to this network, you will enter the equipment's controller panel.

The following is an image of the controller menu:

Note: It is likely that depending on the cell phone, you will have to accept the connection and then log in again.

Note: The response speed of the sent signal will likely be faster in some cell phones than in others.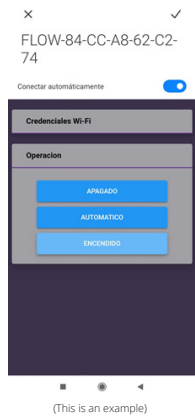

(This is an example)


- 4|You will then have access to the control menu of your equipment.**
The different modes are the same as those executed from the manual push button. See *Push button operating modes*.
- 5|You can enter manually to a menu with more options when you are connected to this network and enter in your browser to the following address:**
<http://192.168.100.50/> or <http://uvflow.local/>

Connecting the UVFlow to your local Wi-Fi internet network

This option is to use the equipment without disconnecting from the internet every time you need to use your equipment.

To operate the equipment in "Local Network Connection" mode, you must:

1|Display the option: "Wi-Fi credentials".

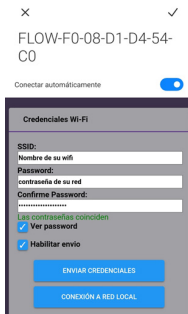SSID: enter the name of your network
Password: Password of your network
Check the options: "Enable Delivery"

Clicking on "Send credentials" will send your network information to the internal memory of your FLOW equipment. You will not see any change of status, only how it deletes the entered data.

2|By clicking on "connect to the local network", the computer will reset. This process may take a few minutes. Internally the equipment is doing the update and matching its IP network to your local network.

3|Reconnect to the FLOW equipment network again (step 1). Exit the initial driver (reduced menu) and enter the extended equipment driver. For this, access the web browser (Chrome, Mozilla, etc.), use the equipment's IP (192.168.100.50), and enter the setup tab to see what local IP was assigned, as shown in the following image:

Note: for step 3, you must still be connected to the network of the Flow unit, for example:

NETWORK connected: FLOW-XX-XX-XX-XX-XX-XX

IP in browser: 192.168.100.50

Controller tab: "Setup".

By following these steps, you will be inside the extended controller of the equipment.

Once you have validated your network credentials, you can now verify which IP was assigned to you.

To do this, you must go to the tab: "Information."

There the COPY IP ADDRESS:


In our example, the assigned address is as follows:

IP ADDRESS:

192.168.0.19

THIS IS AN EXAMPLE. In your equipment will completely change the data presented here. The settings will be sent in the same way.

4|You can now connect to your local network and use this IP in your usual browser (Google Chrome, Mozilla, etc.) to manage your FLOW equipment.

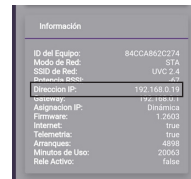Maintenance and repair

Avoid making repairs or adjustments to the equipment; this should be done by trained personnel, who will be in charge of corrective and preventive maintenance.

Information on the collection of tubes after their useful life or if they break:

Tube breakage:

The procedure to be performed is the same as you would perform with a fluorescent tube.

1|If a UVC fluorescent tube breaks, the first thing to do is not to touch the mercury dust under any circumstances.

2|Avoid direct contact with the materials. The person in charge of cleaning should always use an external object to protect his/her skin (gloves, paper, etc.).

3|Once the broken glass has been collected, clean the heavy metal with a damp cloth.

4|Store the damp cloth with the mercury powder in a sealed container.

⏪ Maintenance and repair

Warning: It is advisable to avoid brooms, vacuum cleaners, or other equipments, as the mercury powder adheres easily to any surface.

Warning: Do not throw directly into the glass container or any other recycling container. The bulbs should be taken to a clean point or green point to be processed there with specialized machines. Consult your local area.

Shelf life of the tubes:

Contact Terragene® or one of its representatives to purchase a replacement lamp at the end of their useful life.

Instructions for changing the germicidal tube

Warning: This operation must always be performed by adults. The equipment must be disconnected from the power supply during the tube change.

Steps:

- 1|** Ensure that the UVFlow equipment is switched off and with the power cord disconnected from the mains.
- 2|** Place the equipment on a horizontal surface with the curved front (*with the Terragene® logo*) facing upwards.
- 3|** Remove the six cross-type fixing screws located on both sides of the curved front.
- 4|** Remove the curved front of the unit. Remember, it can only be removed if the unit is switched off and disconnected from the mains.
- 5|** Lightly pull the upper T8 socket and remove the germicidal UVC tube to be replaced by holding it by the white head and gently pulling it towards the lower socket.
- 6|** If both tubes blow, repeat Step 5 for the other germicidal UVC tube.
- 7|** Fit the new germicidal UVC tube(s) by first locating the pins in the lower socket and then the other pins by gently pulling the upper socket.
- 8|** Fit the curved front.
- 9|** Insert the six fixing screws. Remember **never** to connect the equipment to the power supply if the protective cover and its fixing screws are not in place.
- 10|** Connect the equipment to the power supply and switch it on to protect people from dangerous pathogens.

■ Transportation and storage

Delivery inspection

Packing inspection:

- 1|** Check the package and check that it is not damaged.
- 2|** Compare the parts with those listed on the receipt and the shipping slip, and check that none is missing or damaged.
- 3|** Claim to the shipping company if there is a problem.

If the product was collected from a distributor, please complain directly to the distributor.

Unit Inspection

- 1|** Remove the packaging materials from the product. Dispose of the packaging materials according to local regulations.
- 2|** Inspect the product for damaged or missing parts.
- 3|** If applicable, disassemble the product by removing screws, bolts, or bands. For your safety, be careful when handling nails and bands.
- 4|** If any damage is found, contact your sales representative.

Transport Guidelines

Handling and Lifting the UVFlow

Keep the unit in the same position in which it was shipped from the factory.

Remove the indicated cover and use the natural displacement of the product to remove it from its packaging.

Risk of crushing. The unit and components may be heavy. Use proper lifting methods and wear steel-toed shoes at all times.

Storage Guidelines

Long-term storage

If the equipment is stored for more than six months, these requirements should apply:

- Store the unit in a dry location.
- Store the unit in a cool place free from dirt and vibration.

↩ Transportation and storage

If you have questions about possible long-term storage treatment services, contact your local Terragene® sales representative.

■ Product warranty

Liability

Terragene® is not responsible for any personal injury, damage, or delay caused by failure to follow the installation, operation, and maintenance instructions and procedures contained in this manual. Under no circumstances should the equipment be used on humans. Disinfection equipment that includes UVC technology must be used only on inanimate surfaces, air, and or water. Buyer accepts responsibility for checking that the equipment is not otherwise used and shall indemnify and hold Seller harmless from all liability (*including liability arising from Seller's negligence*) arising from such misuse.

Warranty Claims

Terragene® products are manufactured under the highest quality standards and have a reliable and long-lasting performance expectation. However, should the need for a warranty claim arise, contact your Terragene® representative.

Coverage and Limitations

Terragene® UVFlow disinfection units are warranted to be free from defects in material and workforce for one year from the date of shipment from the factory in Argentina. The legal or otherwise obligation under this Warranty is limited to:

Coverage

Terragene® agrees to remedy UVC product failures under these conditions:

- The replacement or repair of the part which has material or workforce defects after being inspected by Terragene® or any Terragene® qualified supplier.

- These failures are reported to a Terragene® representative within the warranty period.
- This warranty does not obligate Terragene® to bear the cost of labor or transportation charges in connection with the replacement or repair of defective parts, nor does it apply to equipment on which repairs or alterations have been made unless authorized by Terragene®.
- The product is used only under the conditions described in this manual.
- The monitoring equipment incorporated in the product is properly connected and in use.
- No warranty is made concerning the tablet, or commercial accessories, which are subject to the warranties of their respective manufacturers.
- No express, implied, or statutory warranty is made in addition to that stipulated or authorized by Terragene®.
- In no event shall Terragene® be liable for any damages or contingent liabilities arising out of the failure or damage of any equipment over others or the improper operation of its parts.
- All repair and service work is performed by Terragene®'s authorized personnel.
- Genuine UVC parts are used.

Limitations

The warranty does not cover failures caused by the following situations:

- Deficient maintenance.
- Modifications or changes to the product and installation made without prior consultation with Terragene®.
- Repair work improperly performed.
- Normal wear and corrosion.
- Disassembly of the equipment.

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