

Create a better future









Chemdye® *splat*
RESULTS REFERENCE GUIDE IN
WASHER DISINFECTORS

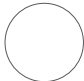

✓ OK
✗ FAIL

BASKET ZONE
influence results

COVERED ZONE
influence results

Visual results	Location inside washer	Water pressure	Mechanical functions (detergent injection pump, spray arms, etc.)	Temperature	Exposure time	Detergent (type, dose)	Water quality
 Processed and correct	✓	✓	✓	✓	✓	✓	✓
	✓	✓	✓	✗	✗	✗	✗
	✓	✓	✓	xx	xx	xx	xx
	✓	✓	✓	xxx	xxx	xxx	xxx
	✗	✗	✗	xxx	xxx	xxx	xxx
	✗	✗	xxx	xxx	xxx	xxx	xxx
	xxx	xxx	xxx	xxx	xxx	xxx	xxx
 Unprocessed	—	—	—	—	—	—	—



IF YOU WANT TO DIGITIZE AND AUTOMATE THE CONTROL AND RECORDING OF WASHING PROCESSES, GET TO KNOW TRAZANTO'S TECHNOLOGY.

