Rev. 24 / 03.2024

Cleaning Performance Test

Chemdy® Splat CDWAH Holder

- (A) BASKET ZONE:** Water pressure, spray system performance, location and other mechanical functions.
- (B) COVERED ZONE:** Detergent chemistry and concentration, water quality, temperature, cycle duration, etc.

ATTENTION: *If your chemical indicator has this logo printed on it, it means that your product is compatible with exclusive Trazanto Automatic Quality Control and Traceability System for cleaning, disinfection and sterilization processes. This System interprets the indicator's results in an accurate and automatic manner, and stores all the process data through Bionova® Software. Use Trazanto scanner to interpret the results (see instructions for use in the manual of the device).*

ATTENTION: *Do not write on the indicator before scanning.*

Chemdy® Splat CDWAH-U Holder

Only for ultrasonic cleaning performance test.

NOTE: Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the recolor as shown in the actual indicators. Images are reproduced for illustration purposes only, and do not necessarily match the article described in every detail.

Results reference guide in Washer-Disinfectors (✓OK / X FAIL)									
BASKET ZONE influence results					COVERED ZONE influence results				
Visual results	Location inside washer	Water pressure	Mechanical functions (detergent injection arms, etc.)	Temperature	Exposure time	Detergent (type, dose)	Water quality		
Processed and correct	✓	✓	✓	✓	✓	✓	✓		
	✓	✓	✓	X	X	X	X		
	✓	✓	✓	XX	XX	XX	XX		
	✓	✓	✓	XXX	XXX	XXX	XXX		
	X	X	X	XXX	XXX	XXX	XXX		
	X	X	XXX	XXX	XXX	XXX	XXX		
Unprocessed	XXX	XXX	XXX	XXX	XXX	XXX	XXX		

NOTE: This reference table corresponds to results when using Chemdy® Splat CDWA4 in Washer-Disinfectors and could be a consequence of failure in one or a combination of variables. When detecting the failed variable/s, make the corrective actions. If the problem persists, contact the technical service of your washing machine. The higher the quantity of **X**, the higher the influence of that variable on the failure of the cycle. Same results could be obtained when using Chemdy® Splat CDWA3 under the user's specific conditions.

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Product description
Chemdy® Splat CDWA indicators consist of a 70 x 30 mm temperature stable support containing a mixture of specially combined natural colored components (proteins, carbohydrates, fatty acids, and dyes). In all cases, indicator colored test soil is non-toxic, dissolves with most of the cleaning agent and is swept along with the washing fluid. Chemdy® Splat CDWA cleaning indicators formula allows detection of all the factors that affect the outcome of the washing process. Chemdy® Splat CDWA indicators can be used for routine control of cleaning processes and allow monitoring of washing effectiveness in different washing machines including ultrasonic washing machines with different washing configurations.

Control of cleaning and washing processes of medical instruments and materials is highly important since the outcome affects subsequent disinfection and/or sterilization processes success. Cleaning control is particularly relevant in automatic washing machines, where even though process parameters may be well controlled, the cleaning final result depends largely on uncontrolled factors. Washing efficacy is influenced by detergent, water temperature, time and washing penetration. A good cleaning result is achieved by an appropriate combination of these variables. The same considerations apply for washing processes in ultrasonic washing machines.

EN Cleaning Indicators
For cleaning performance tests

EN

EN

Indicators Classification

Code	Sample color	Challenge level	Recommended use in thermodesinfectors washing	Recommended use in ultrasonic washing (35-40 kHz) (*)
CDWA3	Blue	High	Temperatura > 40 °C Cicle duration > 3 min.	Temp. 30-50 °C Enz (%) 0.5% ≥ 10 min. Alk (%) 0.2% ≥ 5 min. Temp. 50-70 °C (*) Alk 0.2% ≥ 3 min.
CDWA4	Fuchsia	Very high	Temperatura > 50 °C Cicle duration > 5 min.	Temp. 30-50 °C Enz 0.5% ≥ 20 min. Alk 0.2% ≥ 10 min. Temp. 50-70 °C Alk 0.2% ≥ 7 min.

*1: The tests carried out with Chemdy® Splat CDWA indicators in ultrasonic washing machines were performed using the holder designed for this purpose, Chemdy® Splat CDWAH-U, which allows to fasten the indicator in a specific

location inside the ultrasonic bath, without representing a challenge to the washing processes.

*2: Enzymatic detergent, tested at a standard concentration of 0.5 % V / V.

*3: Alkaline detergent, tested at a standard concentration of 0.2 % V / V.

*4: Products were not tested with enzymatic detergent in the range between 50-70 °C, because most enzymatic detergents may not work efficiently at this temperature range.

NOTE: It is recommendable to test the 2 types of Chemdy® Splat CDWA indicators along with the instrument, in order to select the one which is more appropriate for the washing machine performance. It is advisable to start the washing control validation process using the most rigorous Chemdy® Splat CDWA4 product. The difficult access to the washes, or completely during the washing/disinfection program should be used for routine monitoring. It should be noticed that the choice of indicator will also depend on the hardness of dirt adhered to the object to be washed. Therefore, the Chemdy® Splat CDWA indicator selected for routine monitoring of a particular washing machine could be changed according to the hardness of the soil on the load. Once the Chemdy® Splat CDWA indicator appropriate for the washing requirements has been selected, we recommend placing several identical indicators at different locations in the washing machine, so that we can monitor the performance of all areas of the machine during the washing process.

Characteristics
Chemdy® Splat CDWA indicators have a space for recording information and are placed within a special holder (Chemdy® Splat CDWAH or Chemdy® Splat CDWAH-U holder, see figure) that can be fixed on a tray. Holder design mimics a realistic challenge of thoroughly cleaning hard-to-reach areas to the washing disinfecting process. When Chemdy® Splat CDWA is placed within the Chemdy® Splat CDWAH holder, half of the test soil is obscured (COVERED ZONE in the figure), thus simulating the joint of a surgical instrument, with water stream hitting it indirectly. The other half of the test soil is covered by a mesh (BASKET ZONE in the figure), thus posing a difficulty for water stream to reach it, mimicking shadowed or occluded surfaces.

Chemdy® Splat CDWA indicators provide constant cleaning characteristics compared to the use of natural samples based assays which might suffer modifications of adhesion and polarity properties during storage time. The Chemdy® Splat CDWA system provides high confidence and high reliability to reveal defects in the cleaning process.

Instructions for use

1. Choose the Chemdy® Splat CDWA indicator according to the desired homogeneity of the cleaning process.
2. Place the indicator on the Chemdy® Splat CDWAH or Chemdy® Splat CDWAH-U holder.
3. Place the system on the tray along with the material. The Splat system can also be used within an empty washer (without material).
4. Once the cleaning cycle is over, remove the indicator.
5. Analyze Chemdy® Splat CDWA indicators using the reference table and record results. If unfavorable results are obtained, perform the appropriate corrective actions and repeat the washing process.

Use at least one Chemdy® Splat CDWA indicator for each cleaning cycle.

NOTE: For the holder model in which the test soil is completely covered by a mesh, the effect of washing/disinfecting process is homogeneous, unlike to the test holder design in which only the top half allows appreciating two different effects. Therefore, if the full mesh holder is used, it should be noted that the results shown in the reference table for the top half of the test soil will be repeated in the bottom half.

NOTE: If any serious incident occurs in relation to the device, it should be reported to Terragene S.A. and the competent authority of the State in which the user is established.

Observations

Technical specifications outlined in ISO/TS 15883-5:2005 standards describe different types of test samples having different properties and hardness facing cleaning, but no recommendation is made about which sample type should be used. Currently, none of the samples is suggested as a reference or standard because no method has been defined to compare the properties of the materials used in these tests. For this reason, Terragene® provides two washing indicators with different levels of response and adhesion.

Precautions

Do not reuse Chemdy® Splat CDWA indicators. Do not touch the indicator ink with your hands.

Storage

Chemdy® Splat CDWA indicators should be stored at temperatures between 10-30 °C, protected from light, RH 30-80 %, within its original packaging.

NOTE: In case of remaining traces of indicator colored spot after the washing process, do not store the used indicator under direct light since this may deteriorate its coloring.

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