


Holder for Ultrasonic Cleaning Indicator




Instructions for use


Before the test

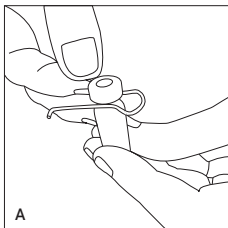


Place the holder around the neck of the capsule.

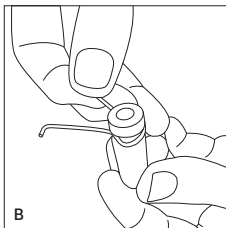

Press the holder down so that the groove of the holder fits securely around the neck of the vial.

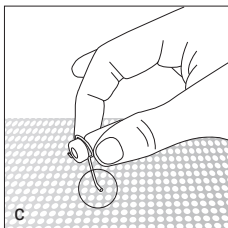


Insert the L shaped end on either side of the holder into the holes of the Ultrasonic device's basket to hold the vial in position. The holder can be attached to a wire basket or perforated basket.

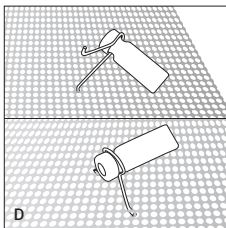


Vials should be placed in the correct location within a tray / basket according to Ultrasonic Cleaning Indicator's IFU based on the size of the tank.

After the test is complete

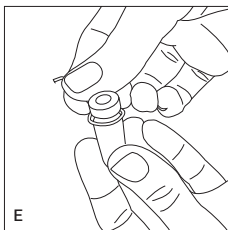

After the ultrasonic cycle remove the vial holding it with three finger of one hand and the index finger and thumb of the other hand on the holder and lid of capsule.

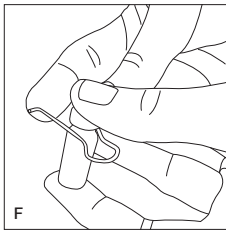

Place the index finger on the end of the holder, then place the thumb on the lid of the capsule for support, push the holder back off the neck of the capsule and remove the holder with both fingers.

NOTE: Dry after use and store in a dry place.