PRO1 Validation Kit

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Explanation of Symbols

- CE mark. Authorized representative in the European Community. Medical Device.
- LOT Batch number.
- Manufacture Date.
- Expiration Date.
- Manufacturer.

Instructions for use

Remove swab

A Remove the swab from the device.

B Submerge for 5 seconds the absorbing part of the swab in the vial's solution.

C Place the swab back into the device.

D Activate.

E Shake downwards vigorously until the solution turns to green and reaches the readout cone reference volume.

F Shake for 15 seconds with the swab inside the readout cone.

G Slide swab upwards without completely removing.

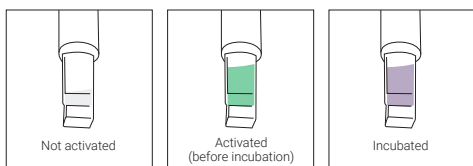
H Again, shake downwards vigorously and reach the readout cone reference volume.

I Readout cone without swab.


J Incubate immediately and read.

Results reference guide

Positive Control (Light blue cap vial)

Negative Control (Transparent cap vial)

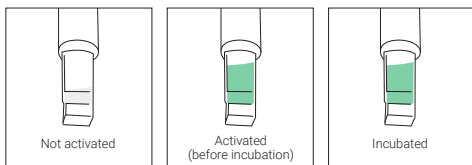NOTE: Reference colors exhibited in printed prospects and boxes as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

EN Validation Kit

For the validation of PRO1 MICRO Hygiene Monitoring System's efficiency

Product description

Chemdye® PRO1 VT Validation Kit has been designed for the validation of PRO1 MICRO Hygiene Monitoring System's efficiency when used with Bionova® MiniPro, IC10/20FR or IC10/20FRLCD incubators. It is capable of detecting proteins and reducing the impact of iatrogenic agents on surgical instruments and surfaces after the cleaning process, in real time. Chemdye® PRO1 VT Validation Kit consists of a vial with a free-protein solution as a negative control (transparent cap) and a vial with a solution containing a certain amount of standard protein as a positive control (light blue cap). The use of these vials allows the validation of Chemdye® PRO1 MICRO protein pen efficiency along with Bionova® MiniPro, IC10/20FR or IC10/20FRLCD incubators.

Indications for use

Use Chemdye® PRO1 VT Validation Kit when you wish to validate the efficiency of Chemdye® PRO1 MICRO Protein Pen using Bionova® incubators.

Precautions

Do not open until use. Do not reuse. Use latex gloves, facial mask and other hygiene measures. Do not freeze.

Instructions for use

IMPORTANT: The instructions for use are the same for both negative and positive control vials. Do not use the moisturizing solution provided along with Chemdye® PRO1 MICRO Protein Pen. The tests have to be carried out with both components at room temperature (20-25 °C).

1. Remove the vials from the kit minutes before the test and allow them to reach a temperature between 20-25 °C. Uncover the vial to be evaluated.
2. Remove the swab from your Chemdye® PRO1 MICRO Protein Pen (Fig. A) and submerge the absorbing part in the vial's solution until the swab reaches the bottom (Fig. B). Keep it there for 5 seconds.
3. Remove the swab from the vial and place it again in the pen (Fig. C).
4. Activate it, by pressing firmly downwards (Fig. D).
5. Shake vigorously downwards, until the solution turns to green and reaches the reference volume of the readout cone (Fig. E). **CAUTION:** If the device is not shaken properly (so as the solution in the internal compartment goes down completely), the results can be invalid.
6. Shake downwards vigorously for 15 seconds, with the swab submerged in the readout cone solution (Fig. F).
7. Slide the swab upwards, without removing it from the device (Fig. G).
8. Shake downwards vigorously and collect the solution in the readout cone (Fig. H).
9. Make sure the swab is outside the readout cone (Fig. I).
10. Incubate the Chemdye® PRO1 MICRO at 60 ± 2 °C for:
 - 4 minutes in Bionova® MiniPro incubators.
 - 7 minutes in Bionova® IC10/20FR or IC10/20FRLCD incubators (Fig. J).

IMPORTANT: The incubation must be done without the swab submerged in the solution. Read the instructions for use before using these devices. **NOTE:** if any serious incident occurs in relation to the device, it should be reported to Terragene S.A. and the competent authority of the State in which the user is established.

Storage

Chemdye® PRO1 VT Validation Kit should be stored at temperatures between 2-8 °C, away from light. It has a shelf life of 24 months when stored refrigerated (2-8 °C).

Disposal

Discard the product after use according to your country's healthcare and safety regulations.

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