
Hygiene Monitoring System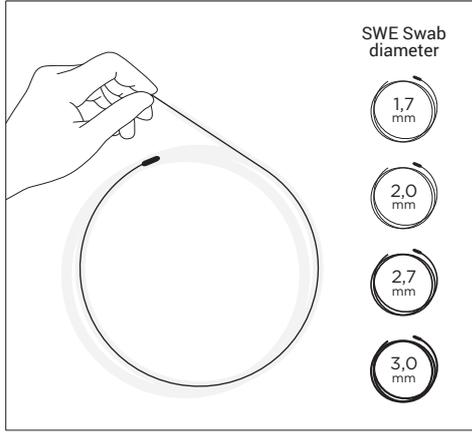

Instructions for use

A

Moisten the swab by immersing it for 5 seconds in the wetting solution

B

Insert the swab into the internal channel of the endoscope or cannulated instrument

C

Cut off the tip of the swab (approximately 20 cm from the end) with scissors

D

Activate the Chemdye® PRO1 ENDO

E

Collect the solution from the inner compartment by shaking vigorously downward until the solution turns green

*Reference volume

F

Remove the handle from the device

G

Gently place the swab for 10 seconds in the solution

H

Slide the swab back up and remove it from the device


I

Place the handle and incubate immediately

J

Pull the cable from the distal end to remove the remaining segment of the swab

Results reference guide

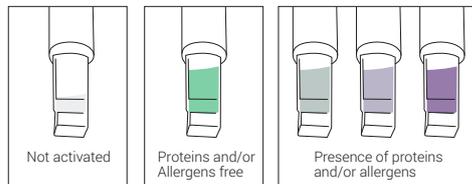NOTE: Reference colors exhibited in printed prospectus and boxes as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

Explanation of Symbols

MD Medical Device.
CE CE mark.
REP Authorized representative in the European Community.

LOT Batch number.
M Manufacture Date.
EXP Expiration Date.
M Manufacturer.

EN Hygiene Monitoring System

To be used with Chemdye® PRO1 ENDO for the detection and quantification of proteins in cannulated instruments

Product Description

The first step in proper cleaning of surgical instruments is to remove blood, body fluids, and tissues immediately after use. If cleaning is not performed properly, the disinfection/sterilization steps will not be effective, leading to the exposure of patients to contaminants such as body fluids, tissues from other patients, and microorganisms.

Chemdye® SWE high-absorption swabs have been specifically designed for use with cannulated instruments. These swabs must be inserted through the internal channel of the cannula, allowing the collection of contaminants that may persist after the cleaning process. SWE swabs should be used with the Chemdye® PRO1 ENDO system, which enables verification of the cleaning of cannulated instruments by detecting protein residues that may remain after the process. The Chemdye® PRO1 ENDO system provides quantitative results in the range of 1 to 50 µg of protein with a limit of detection (LOD) of 0.5 µg*, when incubated in the recommended Bionova® Auto-readers.

The Chemdye® SWE swabs, 2.5 meters in length, have been designed with four different diameters (1.7 mm, 2.0 mm, 2.7 mm, and 3.0 mm), ensuring compatibility with a wide range of cannulated instruments.

*Refers to µg BSA (bovine serum albumin).

Precautions

Do not open until use. Do not reuse. Do not re-incubate. Do not swab surfaces that have not been subjected to a cleaning process or visibly contaminated areas. Avoid touching the absorbent part of the swab or the inside of the device with your hands. Use gloves, a mask, and other hygiene measures. Do not freeze. Do not use Chemdye® SWE swabs on any device other than the Chemdye® PRO1 ENDO. High concentrations of alkaline detergents may cause false-negative results in some cases. Oxidizing disinfectants (such as hydrogen peroxide or peracetic acid) may cause a false-positive color change reaction.

Although Chemdye® PRO1 ENDO is highly sensitive, its intended use corresponds to channel monitoring after the cleaning process, so the test result is not evidence that the device is sterile.

Instructions for use

To access the instructions for the Chemdye® PRO1 ENDO - Chemdye® SWE system, please refer to the attached user instructions in the Chemdye® PRO1 ENDO or scan the following QR code:

Storage

Chemdye® SWE swabs should be stored at temperatures between 15-30 °C, protected from light and relative humidity (RH) of 20-80 %. Their shelf life is 24 months when stored at the declared temperatures.

Disposal

Discard the product after use according to your country's health regulations.

Table of suggested swab diameters according to the internal diameter of the cannula:

Cannula internal diameter (mm)	Chemdye® SWE swab diameter (mm)
2,4	1,7
2,8	2
3,5	2,7
4,5	3

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