## Multivariable Indicators

## Results Reference Guide

## Unprocessed


## Failures

One or more sterilization parameters were not achieved. The materials cannot be considered sterile and must be processed again.

The main failure causes are:

- Inadequate cycle duration.
- Inadequate cycle selection.
- Overloading / incorrect loading of the sterilizer.
- Improper sterilization agent concentration.
- Sterilizer malfunction.
- Blocking indicator with large metal mass.

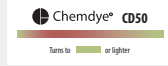## Processed and Satisfactory

**NOTE:** Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

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## Explanation of Symbols

**FORM** Product designed for use in Formaldehyde sterilization processes.

**MD** Medical Device.

**LOT** Batch number.

**CE** CE mark.

**EXP** Expiration Date.

**MAN** Manufacturer.

**EC REP** Authorized representative in the European Community.

**ATTENTION:** If your chemical indicator has this logo printed on it, it means that your product is compatible with exclusive **Trazanto Automatic Quality Control and Traceability System** for cleaning, disinfection and sterilization processes.

This System interprets the indicator's results in an accurate and automatic manner, and stores all the process data through **Bionova® Software**. Use **Trazanto** scanner to interpret the results (see instructions for use in the manual of the device).

**ATTENTION:** Do not write on the indicator before scanning.

## EN Multivariable Indicators

For monitoring Formaldehyde sterilization processes

Type 4 Indicators according to ISO 11140-1:2014

## Product description

Chemdye® CD50 multivariable chemical indicator has been developed to react to Low Temperature Steam Formaldehyde (LTSF) sterilization processes within loads, ensuring an adequate control of the effectiveness of the sterilization processes (temperature, time and gas concentration). Bordeaux-red ink turns to green when the process reaches the stated values of the critical process variables. Multivariable indicators allow to rapidly detect whether the values of the critical parameters of the sterilization processes for which they have been designed to respond, have been reached. Endpoint conditions: 30 minutes, 60 °C, 1 mol/l FORM.

Indicators 100 % free of toxic heavy metals.

## Precautions

Do not expose Chemdye® CD50 multivariable chemical indicator to sterilization processes different from LTSF. Do not wet. Do not reuse Chemdye® CD50 multivariable chemical indicator.

## Instructions for use

1. Place a Chemdye® CD50 multivariable chemical indicator within each package, bag or tray that will be sterilized by LTSF.
2. In rigid containers place a Chemdye® CD50 multivariable chemical indicator in every corner, or at least in two diagonally opposite corners, to ensure that the sterilization agent reaches the most inaccessible areas.
3. Once the process has finished, analyze the results. Chemdye® CD50 multivariable chemical indicator must turn to the reference color for considering the indicated conditions were met.

**NOTE:** If any serious incident occurs in relation to the device, it should be reported to Terragene S.A. and the competent authority of the State in which the user is established.

## Storage and shelf life

Store in a dry place, protected from light, at a temperature between 10-30 °C and at a relative humidity between 30-80 %.

Do not store close to sterilizing agents.

Chemdye® CD50 multivariable chemical indicators have an expiration date of 5 years from the date of manufacture if stored under recommended conditions.

Do not use Chemdye® CD50 multivariable chemical indicators after their expiration date.

Endpoint Stability Reaction: chemical indicator endpoint shall remain unchanged for a period of not less than 6 months when stored at previously indicated conditions.

## Disposal

Discard chemical indicators after use according to your country's healthcare and safety regulations.

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