EN Chemical Indicator

For monitoring disinfection processes of surfaces by means of Quaternary Ammonium

It has been proved that many microorganisms may be inactivated by using quaternary ammonium salts (QUATS). These cationic surfactants interfere with the functions of the bacterial cell membranes and the lipid capsid of the virus, releasing the contents outside the cell, thus rendering them inactive. The minimum doses of quaternary ammonium needed to achieve a certain reduction in the microbial charge depends not only on the particular microorganism but also on additional factors such as the dirt present in the environment, the temperature, the humidity and time of exposure to the chemical agent. Because of the aforesaid, it is of utmost importance to check that all the surfaces to be disinfected receive the same doses of the disinfectant agent. The chemical indicator Chemdye® CD63-ST has been especially developed to satisfy this need. Calibrated to undergo a change in colour only when exposed to Quaternary Ammonium disinfectant salts as active principle with concentrations higher than 0.3 %, it enables to check the efficacy of the disinfection process in a fast and accurate manner.

Product description

The chemical indicator Chemdye® CD63-ST has been developed especially to monitor the surface disinfection processes that use Quaternary Ammonium as disinfectant agent. It consists of a self-adhesive label containing reactive ink and a colour reference for the final colour change. **IMPORTANT:** the reference colour is provided only for illustrative purposes. In addition, the technology of the label enables its traceability by QR code, which allows its use in the DCS disinfection certification system. The traceability QR code for the Disinfection Certificate provides information about the status of the label, indicating whether it has already been used in a Disinfection Process, the results, the date of the disinfection operator, the device used and the place, date and time in which it was subject to the procedure.

Characteristics

29 x 22.5 mm self-adhesive label, printed with indicator ink, for use in the Chemdye® CG3-N manual labeller.

Indicadores 100% free from toxic heavy metals.

Smart Disinfection Certification System (DCS)

The CD63-ST labels for use with Quaternary Ammonium salts disinfectants have been designed as part of a brand new Certification Disinfection System. Thanks to its exclusive design, the DCS (Disinfection Certification System), enables the systemization and recording of the disinfection process by using the traceability of the CD63-ST label. Through its DCS App, the DCS uses a criterion based on artificial intelligence (IA) to guide the disinfection system that includes both good practices suggestions and standardised disinfections protocols and makes it possible to interpret the result of the exposure to the disinfectant agent through direct reading with a Smartphone. For further information, please visit www.terragene.com/dcs-mobile-app/.

Instructions for use

1. Place the Chemdye® CD63-ST indicator on the surface where the disinfection level is to be checked. Use as many indicators as surfaces need to be verified, according to the DCS protocol available in the DCS app. **NOTE:** Recommended use: 4 Chemdye® CD63-ST chemical indicators per m². The indicators should be placed not higher than 1.5 m above ground level. 2. Run a normal disinfection cycle. **IMPORTANT: the label was designed to be used only with Quaternary Ammonium (QUAT). Its exposure to other substances may deteriorate it or cause it to change to final colours not contemplated for its use.** 3. Once the process is completed, wait at least 15 minutes and compare the final colour of each Chemdye® CD63-ST chemical indicator against the reference strips placed on and under the reactive ink. The final colour depends on the concentration of Quaternary Ammonium salts to which the reactive ink was exposed. For higher concentrations the label shows a greenish-blue colour. If the colour of the reactive label is equal or darker and more bluish than the reference, the disinfection process has been successful. The graph and table attached allow the rapid recording and analysis of the results obtained. **IMPORTANT: the label must be analyzed after the first 15 minutes and before 3 hours after exposure to the disinfecting agent.** 4. To analyse the App DCS app, make sure to air dry the remaining moisture or drops that could have been deposited on the indicator before photo capturing it with your Smartphone. **IMPORTANT: the records of the exposed Chemdye® CD63-ST should be protected against humidity.**

Labelling using the CG3-N manual labeller

To apply the indicator label using a Chemdye® CG3-N labeller, follow the following instructions carefully:

A. Insert the label roll in the Labeller

1. Press the top button (Fig. 1) and open the cover by lifting it completely

until you hear a click. Do not force the cover upwards once you have heard the click. 2. To load the label roll, unroll the first 10 Chemdye® CG3-N labels, detach and discard them making sure only the liner remains (Fig. 2). 3. Insert the label roll between the fasteners as shown in Fig. 3. The end of the roll must be placed right side up. The roll must be centred between the fasteners. 4. Pull the end of the roll and centre it on the guide rod (Fig. 4). Pull the end of the roll until 15 and 16 cm emerge. 5. Close the cover (Fig. 5). Make sure to hear the plastic parts click when the cover is fully closed. 6. Hold the trigger and insert the end of the liner into the slot below the grey cylinder until it goes no further (Fig. 6). 7. The liner will be loose, adjust to correct tension by pulling and releasing the trigger several times. The end of the liner should emerge from the slot located on the back of the labeller (Fig. 7).

B. Labelling of the surface to be disinfected:

Once the label roll has been inserted everything is ready to start labelling the surfaces. 1. Pull and release the labeller's trigger as many times as necessary until the label comes out through the exit slot (Fig. 8). Pull the trigger strongly until it moves no further to prevent the liner from running incorrectly and to ensure that the label comes out in the correct position. 2. Place the labeller vertically against the surface to be labelled (Fig. 9). 3. Prop the exposed label on the surface to be labelled. The printed side of the label must be visible and the adhesive side must make contact with the surface. 4. Slide the labeller so that the label is stuck on the surface until it is fully exposed (Fig. 10). 5. The empty liner will gradually come out of the slot on the rear part of the labeller (Fig. 11). Cut the liner when its length hinders the labelling process. 6. Repeat steps 1 to 4 for each surface to be labelled.

C. Chemdye® CG3-N cleaning and fixing jams:

1. Press the top black button (Fig. 12) and open the cover by lifting it until you hear a click. Do not force the cover upwards once you have heard the click. 2. Remove the label roll from the fasteners by sliding the roll upwards (Fig. 13). 3. Place the roll at the same level as the grey cylinder (Fig. 14). 4. Cut the tape near the grey cylinder and place the removed roll in a safe place, away from the sterilizing agent. 5. Remove the liner remnant from the slot located on the back of the labeller sliding it away from the labeller (Fig. 15). 6. Remove the loose labels that remain inside the label and under the guide rod (Fig. 16). 7. Use an adhesive remover (preferably isopropyl alcohol) and a cotton swab to remove any adhesive residue. 8. Soak the swab in the solvent avoiding any excessive absorption. 9. Gently rub the swab against the surfaces or bearing surfaces where solvent residues are observed (Fig. 17). 10. Wait approximately 1.5 minutes until the solvent evaporates and leaves the plastic surface. **IMPORTANT: make sure to remove all solvent residue. If solvent residue comes into contact with the reactive labels, the labels may be damaged.** 11. Close the cover (Fig. 18). Make sure you hear the plastic parts click when the cover is fully closed.

D. Cleaning and clearing the jammed liner from the exit slot of the Chemdye® CG3-N:

In the event that during the labelling process the empty liner coming out of the exit slot located on the back of the labeller gets jammed: 1. Stop the labelling process. 2. Press the top button (Fig. 19) and open the cover completely until you hear a click. Do not force the cover upwards after you hear the click. 3. Remove the label roll from the fasteners by sliding the roll upwards (Fig. 20). 4. Place the roll at the same level as the grey cylinder (Fig. 21). 5. Cut the tape near the grey cylinder and place the removed roll in a safe place, away from the sterilizing agent. 6. Close the cover (Fig. 22). Be sure to hear the plastic parts click when fully closed. 7. Press the labeller trigger while the liner remnant is removed to make sure that the label has been fully cleared (Fig. 23). 8. Visually check that no liner remains inside the labeller before it is reused. If remnants persist, shake the labeller (pressing and releasing the trigger) until the remnants falls as shown in Fig. 24.

Precautions

Use only with disinfectants based on Quaternary Ammonium salt. The use of disinfectants whose formula contain any other component will produce wrong results. Do not reuse the chemical indicators. **Do not expose to products containing other chemical agents (e.g. 2-phenylphenol, ethanol, etc.). The exposure to products other than QUAT may cause colour to change to deep blue and/or the shedding of the reactive, altering it.**

Product storage and expiration date

Store the Chemdye® CD63-ST in the bag in a dry place, away from light, at a temperature between 10 °C and 30 °C, at a relative humidity between 30 % and 80 %. Do not wet. Do not store near disinfectant/sterilizing agents. If stored as recommended, indicators expire 5 years from the date of manufacture. Do not use the Chemdye® CD63-ST chemical indicator after its expiration date. If the indicators are not used over a long time, the indicator should be removed from the automatic labeller and placed back into the original bag.

Waste treatment

Discard the used chemical indicators in accordance with your country's sanitary regulations.

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