## Unique Point Emulator Indicator

IT27-3YS | IT27-4YS | IT27-5YS | IT27-7YS | IT27-18YS

## Result Reference Guide


Unprocessed

Failures

One or more parameters of sterilization were not achieved. The materials cannot be considered sterile and must be processed again. These results are usually obtained when steam quality is poor or adequate vacuum has not been generated.

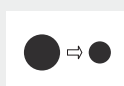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NOTE: Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

## Explanation of Symbols

 Product designed for use with Steam sterilization cycles.

 Batch number.

 Manufacture Date.

 Expiration Date.

 Manufacturer.

 CE mark.

 Authorized representative in the European Community.

## EN Unique point Emulator Indicators

For Steam sterilization processes

Type 6 Indicator according to ISO 11140-1:2014

## Product description

Integron® Indicator Emulators are designed for internal use in steam sterilization processes, for ensuring the achievement of all the critical variables (temperature, time, steam quality). The yellow ink was developed to change to black when the process reaches the stated values (SV) of the critical process variables indicated in the emulator.

## Characteristics

Integron® Emulator Indicators IT27-3YS, IT27-4YS, IT27-5YS, IT27-7YS and IT27-18YS consist of a paper strip of 70 x 23 mm.

Indicators 100 % free of toxic heavy metals.

## Indications for use

Use Integron® Emulator Indicators for monitoring Steam sterilization processes.

## Emulation condition:

IT27-3YS	3 min.	134 °C
IT27-4YS	4 min.	134 °C - 12 min. 121 °C
IT27-5YS	5 min.	134 °C - 15 min. 121 °C
IT27-7YS	7 min.	134 °C - 20 min. 121 °C
IT27-18YS	18 min.	134 °C

## Precautions

Do not expose Integron® Emulator Indicators to EO, Dry Heat, or any sterilization processes other than Steam. Do not use the sterilizer until the emulator's colour change matches the emulation condition.

## Instructions of use

- Place an Integron® Emulator Indicator within each package, bag or tray that will be sterilized by Steam.
  - In rigid containers place an emulator indicator in every corner, or at least two diagonally or opposite corners, to ensure that the steam has reached the most inaccessible areas.
  - Finished the process, analyze the results. The indicator must change to the reference color for considering that emulation conditions were reached.
- NOTE:** if any serious incident occurs in relation to the device, it should be reported to Terragene S.A. and the competent authority of the State in which the user is established.

## Storage and shelf life

- Indicators should be stored in dry places, protect from light at temperatures between 10-30 °C, 30-80 % relative humidity. Do not wet. Do not store close to sterilizing agents.
- Indicators have an expiration date of 5 years from the date of manufacture when stored in the recommended conditions. Both expiration date and batch of the product are specified on the package label and on each unit of product. Do not use indicators after their expiration date.
- After use, the indicator shall remain unchanged for a period of not less than 6 months when stored at temperature and relative humidity previously indicated.

## Disposal

Discard along with cardboard waste according to your country's healthcare regulations.

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