# Indicador integrador

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Class 5 Indicator according to EN ISO 11140-1:2009

#### Product description

IT30 indicator integrator was developed to react in dry heat sterilization processes between 160 °C and 180 °C. The blue ink was fixed to change to brown when the population of spores is completely dead, indicating integration condition is reached. This condition is calibrated with the time of death of Bionova® biological indicators, manufactured according to EN ISO 11138.

#### Characteristics

Consists of a paper strip of 60 mm x 25 mm.

Indicators 100 % free of all metals.

#### Indications for use

Use the indicator integrator to monitor dry heat sterilization processes. Integration condition is reached in dry heat sterilization processes as described in the quality certificate.

#### Precautions

Do not expose this product to EO, steam, or any chemical or physical agent sterilization processes other than dry heat. Do not reuse the sterilizer until the integrator's color reaches or passes the integration condition.

#### Instructions of use

- Place an indicator integrator within each package or tray that will be sterilized by dry heat.
- After completing the process, analyze the results.

#### Interpretation of the results

If the integration integration condition has been reached or exceeded, the sterilization process was successful.

Initial color: **blue**  
Final color: **brown**

#### Storage

- Indicators should be stored in dry places, protect from light at temperatures between 10-30 °C, 30-80 % relative humidity.
- Do not wet.
- Do not storage close to sterilizing agent.
- Do not reuse.

#### Shelf life

- Indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions. The expiration date as the batch of the product are specified on the package label and in each unit of product.
- Do not use indicators after their expiration date.
- After use, the indicator will not change in 1 year if stored in temperature and relative humidity previously indicated.