
## Result reference guide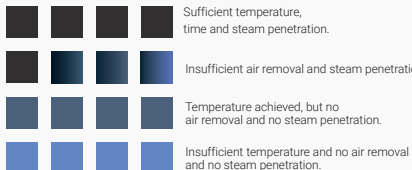

Sufficient temperature, time and steam penetration.

Insufficient air removal and steam penetration.

Temperature achieved, but no air removal and no steam penetration.

Insufficient temperature and no air removal and no steam penetration.

**NOTE:** Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

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### Explanation of Symbols

Product designed for use with Steam sterilization cycles.

CE mark.

Authorized representative in the European Community.

Medical Device.

Batch number.

Expiration Date.

Manufacturer.

## EN PCD-A-3.5BD

For monitoring steam sterilization processes

Bowie-Dick test simulation for use with Helix-PCD system to control steam sterilization processes (Type 2 Chemical Indicator according to ISO 11140-1:2014).

*Indicators 100 % free of toxic heavy metals.*

### Instructions for use

1. Insert PCD-A-3.5BD chemical indicator, according to the selected process conditions, within the pin (metal capsule) or slot (plastic capsule) of the screw cap, settle the reactive indicating ink towards the capsule lumen, avoiding folding those areas printed with the reactive indicating ink.

**IMPORTANT:** avoid contact between PCD-A-3.5BD chemical indicator and capsule interior surface, because it can alter the result.

2. Put the capsule cap with moderate pressure.

3. Settle the system into the fabric bag.

4. The entire system should be sterilized in a vacuum chamber under conditions that depend on the chemical indicator to be used.

**PCD-A-3.5BD Bowie-Dick cycle 3.5 min. 134 °C**

5. Remove the indicator and evaluate the result. If the 4 bars change from blue to dark grey/black, the result can be considered successful, which means the sterilizer is working correctly. If one bar doesn't reach the final color, it means that residual air is present in the sterilizer, therefore a malfunction of the equipment.

The reasons for the failure can be insufficient air removal, leakages and noncondensable gases in steam. We suggest to perform the test once again, before sending the equipment to technical service.

### Storage

Store in a dry place, protected from light, at a temperature between 10-30 °C, and at a relative humidity between 30-80 %. Do not wet.  
 Do not store close to sterilizing agents.

### Shelf life

PCD-A-3.5BD chemical indicators have an expiration date of 5 years from the date of manufacture when stored under recommended storage conditions. Both, expiration date and batch number of the product, are specified on the package label and in each product unit.

Do not use PCD-A-3.5BD chemical indicators after their expiration date.

Do not reuse.

Endpoint reaction stability: chemical indicator endpoint shall remain unchanged for a period of not less than 6 months when stored at previously indicated conditions.

### Disposal

Discard used chemical indicators with paper waste, according to your country's healthcare and safety regulations.

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Terragene S.A