PCD-A-3.5 PCD-A-5.3
PCD-A-7.0 PCD-A-9.0 PCD-A-18

ES

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EN

For Steam sterilization

Chemical Strips (Type 2 Chemical Indicator according to ISO 11140) for use in Helix-PCD for Steam sterilization processes.

Instructions for use

1. Insert the chemical indicator according to the selected process conditions within the pin (metal capsule) or slot (plastic capsule) of the screw cap; settle the reactive ink toward the capsule lumen, avoiding the folds into the chemical ink. **IMPORTANT:** Avoid the contact between the chemical indicator with the capsule interior surface, because result can be altered.

2. Put the capsule cap with a moderate pressure.

3. Settle the system into the fabric bag.

4. The entire system should be sterilized in a vacuum chamber under conditions which depend on the chemical indicator to be used:

PCD-A-3.5 3.5 min. 134 °C

PCD-A-5.3 5.3 min. 134 °C

PCD-A-7.0 7.0 min. 134 °C

PCD-A-9.0 9.0 min. 134 °C

PCD-A-18 18 min. 134 °C

5. Remove the indicator and evaluate the result. If the 4 bars changed from purple to green, the result is successful, the sterilizer is working correctly. If one bar is different to the final color, it shows the presence of residual air in the sterilizer, therefore an equipment malfunction. The reasons for the failure can be air insufficient removal, leakages and steam not condensable gases. We suggest retrying the test once again, before sending the equipment to technical service.

Color change of STEAM indicator strip:

	Sufficient temperature, time and steam penetration.
	Insufficient air removal and steam penetration.
	Temperature achieved, but no air removal and no steam penetration.
	Insufficient temperature and no air removal and no steam penetration.

Storage

Indicators should be stored in dry places, protect from light at temperatures between 10-30 °C, 30-80 % Relative Humidity.

Do not wet.

Do not store close to sterilizing agents.

Do not use indicators after their expiration date.

Do not reuse.

NOTE: Reference colors exhibited in printed prospects as well as those displayed in our website and soft copies of other documents, do not necessarily represent the real color as shown in the actual indicators.

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