BT400

Spore Strip Biological Indicator

For Liquid Peracetic Acid High Level Disinfection.

Usage

Monitoring the sporicidal activity of Liquid Peracetic Acid in High Level Disinfection processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017 and IRAM 37102-1:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.


Characteristics

25.0 mm x 70.0 mm envelope of non-absorbent material. Turquoise code.

5.0 mm x 25.0 mm polyethylene fiber strip.

10^6 *Geobacillus stearothermophilus* ATCC 7953 spores per strip.

Silicone fastener: 80.0 mm long. Pearl end: 5.6 mm diameter.

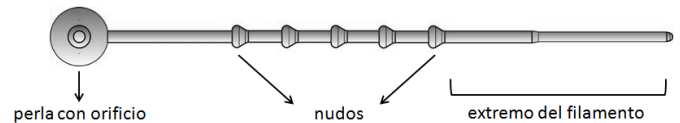Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium at 60°C. We suggest the use of BIONOVA® MC1020 culture media to final reading in 24 hours. For conventional culture media like TSB, is recommended incubation for 7 days at 60 °C.

D-value: at 50°C, 3.500 ppm of Peracetic Acid.

Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH 30-80 %.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

2 years.

Packing

100 envelopes per aluminum foil bag.

Packing information: product description, regulation, storage conditions, manufacturer information and data on packing's label.

Labelling

On product's envelope: product code and description, bacterial strain, batch number, expiration date and datamatrix code.

On product's packing: product code and description, process for intended use, presentation, bacterial strain and load, batch number, manufacture date, expiration date, bar code and datamatrix code.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 60 °C in BIONOVA® IC10/20 incubator.

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to radiation, EO or any processes other than Liquid Peracetic Acid.

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