Infection Control Division | **Biological Indicators**

BT60 Spore Strip Biological Indicator

For ethylene oxide, steam and dry heat sterilization

Usage

Monitoring ethylene oxide, steam and dry heat sterilization processes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016 / EN ISO 13485:2016.

ISO 11138-1:2017, ISO 11138-2:2017, ISO 11138-3:2017, ISO 11138-4:2017 and ISO 11138-8:2017.

Authorization

ANMAT (Argentinian National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Classification

Class 1, according to risk (ANMAT).

Characteristics

Envelope: 25 mm x 70 mm non-absorbent material package, violet edges.

Strip: 5 mm x 25 mm filter paper strip.

10^6 *Geobacillus stearothermophilus* ATCC 7953** and 10^6 *Bacillus atrophaeus* ATCC® 9372 spores per strip.

Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium at 37 °C or at 60 °C. Terragene S.A. recommends the use of Bionova® MC1020-2 or Bionova® MC1030-2 culture media for final reading in 48 hours. For conventional culture mediums, such as TSB, incubation for 7 days at 37 °C or at 60 °C is recommended.

D-Value_{STEAM}: Not lower than 1.5 minutes at 121 °C. D-values at 132 °C and 135 °C are also informed.

Z-Value_{STEAM}: Not lower than 6 °C.

D-Value_{EO}: Not lower than 2.5 min. Conditions: 600 mg/l EO (100 %), RH = 60 %, T = 54 °C.

D-Value_{DRY}: Not lower than 2.5 minutes at 160 °C.

Z-Value_{DRY}: Not lower than 20 °C.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH = 30-80 %.

*ATCC® is a registered trademark of American Type Culture Collection.

Infection Control Division | **Biological Indicators**

BT60 Spore Strip Biological Indicator

For ethylene oxide, steam and dry heat sterilization

Transport conditions

Storage conditions should be followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

Shelf life

2 years.

Packing

100 units per aluminum foil bag.

Labelling

On bag's label: Product code and description, process for intended use, bacterial strain and load, batch number, manufacture and expiration date, presentation, regulation, storage conditions, manufacturer information, barcode and Data Matrix.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 37 °C or 60 °C in Bionova® IC10/20 incubator.

Read product's Instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not use this product for monitoring formaldehyde, vaporized hydrogen peroxide or other sterilization processes different from the specified.

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