Infection Control Division | **Biological Indicators**

## BT92

Spore strips for Plasma or Vaporized Hydrogen Peroxide sterilization processes.

### Usage

Monitoring Plasma or Vaporized Hydrogen Peroxide (H<sub>2</sub>O<sub>2</sub>) Sterilization Processes.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.  
ISO 11138-1:2017 and IRAM 37102-1:1999.

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

### Classification

Class 1, according to risk (ANMAT).

### Characteristics

25 mm x 70 mm non-absorbent material package. Blue code.

5 mm x 25 mm polyethylene fibers strip.

10<sup>5</sup> or 10<sup>6</sup> *Geobacillus stearothermophilus* ATCC 7953 spores per strip.

Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium. The functional and allowed incubation temperature range for Bionova® BT92 is between 55-62 °C. We suggest the use of Bionova® MC20-2 or MC1020-2 culture media to final reading after 24 hours incubation between 55-62 °C. For conventional culture media like TSB, incubation for 7 days at 60 °C is recommended.

### Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

### Storage conditions

T = 10 – 30 °C, RH 30-80 %.

### Transport conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

### Shelf life

2 years.

### Packing

100 units per aluminium foil bag.

Infection Control Division | **Biological Indicators**

## BT92

Spore strips for Plasma or Vaporized Hydrogen Peroxide sterilization processes.

### Labelling

On product's envelope: product code and description, process for intended use, bacterial strain, batch number, expiration date and datamatrix code.

On product's packing: product code and description, process for intended use, strain and population of bacteria, batch number, manufacture and expiration date, presentation, regulation, storage conditions, manufacturer information, barcode and datamatrix code.

### Possible target markets

Healthcare and Industry.

### Other important information

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator. Read product's instructions for use thoroughly before use.

### Precautions

Do not store the product near sterilizing agents.

Do not expose this product to radiation, EO or any sterilization processes other than H<sub>2</sub>O<sub>2</sub>.

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