BTM40G-6

Spore Mini Strips Biological Indicator

For Ethylene Oxide and Dry Heat sterilization processes, in single non-absorbent material envelopes.

Usage

Monitoring Ethylene Oxide (EO) and Dry Heat Sterilization Processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016. ISO 11138-1:2017, ISO 11138-2:2017 and ISO 11138-4:2017; IRAM 37102-1:1999 and IRAM 37102-2:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

20.0 mm x 38.0 mm non-absorbent material envelope. Green code.

2.0 mm x 10.0 mm filter paper mini strip.

10^6 *Bacillus atrophaeus* ATCC 9372 spores per mini strip.

Upon completion of the sterilization cycle, the spore mini strip should be tested in a suitable culture medium at 37 °C. We suggest the use of BIONOVA® MC1020 or MC1030 culture media to final reading in 48 hours. For conventional culture media like TSB, incubation for 7 days at 37 °C is recommended.

D-Value_{DRY}: not lower than 2.0 min at 160 °C.

Z-Value_{DRY}: is informed.

D-Value_{EO}: not lower than 2.0 min, conditions: 600 mg/l EO, 60 % RH, 54 °C for EO.

Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH 30-80 %.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

2 years.

Packing

100 units per aluminum foil bag.

Packing information: product description, regulation, storage conditions, manufacturer information and data on packing's label.

Labelling

On product's envelope: product code and description, bacterial strain, batch number, expiration date and datamatrix code

On product's packing: product code and description, process for intended use, presentation, bacterial strain and load, batch number, manufacture date, expiration date, barcode and datamatrix code.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 37 °C in BIONOVA® IC10/20 incubator.

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to Radiation, H₂O₂ or any sterilization process other than EO and/or Dry Heat.

Note: Spore mini strips can be incubated in BIONOVA® MC1020 or MC1030 culture media or other culture medium made up of soybean casein and yeast extract.

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