Infection Control Division | **Biological Indicators**

KBT400 Spore Strip Biological Indicator Kit

For Liquid Peracetic Acid High Level Disinfection processes.

Usage

Monitoring the sporicidal activity of Liquid Peracetic Acid in High Level Disinfection processes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017 and IRAM 37102-1:1999.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Classification

Class 1, according to risk (ANMAT).

Characteristics

The Kit consists of Spore Strip Biological Indicators Bionova® BT400 and culture medium Bionova® MC1020-2.


BT400. Spore Strip Biological Indicator for Liquid Peracetic Acid High Level Disinfection.

25.0 mm x 70.0 mm envelope of non-absorbent material. Turquoise code.

5.0 mm x 25.0 mm polyethylene fiber strip.

10⁶ *Geobacillus stearothermophilus* ATCC 7953 spores per strip.

Silicone fastener: 80.0 mm long. Pearl end: 5.6 mm diameter.

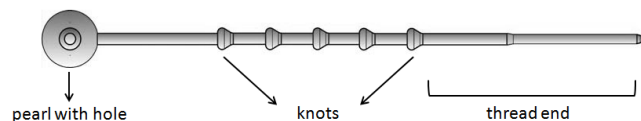Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium at 60 °C. We suggest the use of BIONOVA® MC1020-2 culture media to final reading in 24 hours. For conventional culture media like TSB, incubation for 7 days at 60 °C is recommended .

D-value: at 50 °C, 3.500 ppm of Peracetic Acid.

MC1020-2. Culture medium with pH indicator for growth of *Geobacillus stearothermophilus* ATCC 7953, *Bacillus subtilis* ATCC 35021, *Bacillus pumilus* ATCC 27142 and *Bacillus atrophaeus* ATCC 9372 bacterial spores.

Growth medium color: red.

Plastic tube: 76.0 mm high, 12.5 mm diameter.

Medium volume: 2.0 ml.

Medium turns to yellow when spores grow.

Infection Control Division | **Biological Indicators**

KBT400 Spore Strip Biological Indicator Kit

For Liquid Peracetic Acid High Level Disinfection processes.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10 – 30 °C (BT400), 4 - 20 °C (MC1020-2), RH 30-80 %, keep in dark place.

Transport conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

Shelf life

2 years.

Packing

100 spore strip envelopes + 100 culture medium tubes per box.

Packing information: product code and description, bacterial strain, process for intended use, regulation, storage conditions, manufacturer information and data on the packing's label.

Labelling

On product's envelope: product code and description, bacterial strain, batch number, expiration date and datamatrix code.

On culture medium tubes: product code and description, batch number, expiration date.

On product's packing: product code and description, batch number, Biological Indicator and culture medium batch numbers, expiration date, bar code and datamatrix code.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator.

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing or disinfectant agents.

Do not expose this product to radiation, EO or any processes other than Liquid Peracetic Acid.

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