MC20-2

Culture medium with pH indicator.

For growth of *Geobacillus stearothermophilus* ATCC7953 bacterial spores.

Usage

To test growth of *Geobacillus stearothermophilus* ATCC 7953 bacterial spores used for monitoring Steam, Plasma or Vaporized Hydrogen Peroxide and Formaldehyde sterilization processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016. ISO11138-1:2017, ISO11138-3:2017 and ISO11138-5:2017; IRAM 37102-1:1999 and IRAM 37102-3: 1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

Sterile growth medium is specially formulated for use with Bionova® products (also suitable for other brand products).

Growth medium color: purple.

Plastic tube: 76.0 mm high, 12.5 mm diameter

Medium volume: 2.0 ml

For germination and growth of *Geobacillus stearothermophilus* ATCC 7953 bacterial spores.

Incubation conditions: 24 hours for spores subjected to Steam/VH₂O₂ processes and 48 hours for Formaldehyde processes (maximum). The functional and allowed incubation temperature range for *Geobacillus stearothermophilus* is between 55-62 °C. Incubation at 60 °C is recommended.

Medium turns to yellow when spores grow.

Environmental conditions during manufacture

T= 15-30 °C, RH= 30-80 %. Sterility conditions are necessary during plastic tube filling with sterile medium, performed in laminar flow.

Storage conditions

T= 4-20 °C, RH= 30-80 %, keep out of direct light.

Transport conditions

Storage conditions should be strictly followed.

Transport in closed and reinforced boxes in order to avoid damages.

Transport of this product does not represent a risk for health.

Shelf life

2 years.

Packing

50 units per box.

Packing information: product description, storage conditions, presentation, regulation, manufacturer information and data on box's label.

Labelling

On product: Label 17.0 mm x 42.0 mm printed in black. Code and product description, intended use, batch number and expiration date.

On product's packing: code and product description, intended use, color change, presentation, batch number, manufacture date, expiration date, bar code and data matrix code.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator.

Precautions

Do not store the product near sterilizing agents.

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