## Trazanto

Automatic system for quality control and traceability of cleaning, disinfection and sterilization processes

### Usage

For readout and detection of:

- Chemdye® Trazanto splat CDWA4 cleaning indicator for monitoring automatic and ultrasonic washing machines effectiveness.
- Chemdye® Trazanto Lumenia CDWE cleaning indicator to be used with Chemdye® Lumenia devices to evaluate automatic reprocessing washing machines.
- Integron® Trazanto IT26-1YS Unique point Chemical Integrator for monitoring Steam sterilization processes.
- Integron® Trazanto IT26-C Moving Front Chemical Integrator for monitoring Steam sterilization processes.
- Integron® Trazanto IT26SBL Unique point Chemical Integrator for monitoring Steam sterilization processes.
- Chemdye® Trazanto CD42 Process Indicator for monitoring Hydrogen Peroxide sterilization processes.
- Chemdye® Trazanto CD40 Multivariable Process Indicator for monitoring Hydrogen Peroxide sterilization processes.
- Chemdye® Trazanto CD50 Multivariable Process Indicator for monitoring Formaldehyde sterilization processes
- Chemdye® Trazanto BD125X/1 Bowie-Dick test pack to detect inadequate air removal and steam penetration in vacuum-assisted Steam sterilizers.
- Chemdye® Trazanto BD125X/2 Bowie-Dick test pack to detect inadequate air removal and steam penetration in vacuum-assisted Steam sterilizers.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

IEC 60950-1:2005/A1:2009/A2:2013

RoHS Directive 2011/65/EU

### Characteristics

Trazanto is a system comprised by two main modules: a hardware module composed by the Trazanto Optical Reader, and a software-implemented Analysis Module.

Trazanto Optical Reader

- Dimensions: 270 x 270 x 40 mm
- Voltage range: 100 – 240 V AC
- Frequency: 50-60 Hz
- Power: 18 W
- USB connection between the Trazanto reader and a general-purpose personal computer.

## Trazanto

Automatic system for quality control and traceability of cleaning, disinfection and sterilization processes

### Analysis Module

- Developed for Microsoft® Windows 7 and later.
- Complete and robust integration with Trazanto optical reader.
- Automatic readout and detection of compatible indicators for cleaning, disinfection, and sterilization commercialized by Terragene®
- Intelligent algorithms developed to optimize the results of indicator readouts.
- Convenient display of digitized data for easy edition and identification of approved and rejected indicators.
- Fully compatible with Bionova® Traceability Software.

### Environmental conditions during manufacture

T = 15-35 °C, RH = 35-85 %.

### Storage conditions

T = 10-30 °C, RH = 30-80 %, keep in a dark place.

### Transport conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages.

The transport of this product does not represent any risk for human health.

### Shelf life

2 years.

### Packing

1 unit per box.

Packing information: product code and description, storage conditions and manufacturer information.

### Labelling

On the product and on product's box: product code and description, storage conditions, batch number, serial number, barcode and manufacturer information.

### Possible target markets

Healthcare, Food, Pharmaceutical and medical devices industries.

### Other important information

Trazanto is fully compatible with Bionova® Traceability Software. The Traceability Software enhances Trazanto's capabilities by storing the performed readings for future use, recording them and allowing them to be traced.

A report of the stored readings can be generated in a quick and straightforward way. The report can be saved in PDF format or can be printed to generate a physical copy of the data.

## Trazanto

Automatic system for quality control and traceability of cleaning, disinfection and sterilization processes

### Precautions

Do not pour any liquid inside.

Do not immerse into any liquid.

Only for indoor use.

Disconnect the power cord before cleaning.

Do not use abrasive or corrosive cleaners or disinfectants.

Make sure that the Trazanto Reader is connected to a properly rated power cord.

Unplug the device if you are not going to use it for a prolonged time.

Use only the power adapter and the AC cable included with the device.

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