# CDWAH

## Cleaning indicators holder

For Chemdye® CDWA and Integron® IT27W cleaning indicators

### Usage

Designed to hold Chemdye® CDWA and Integron® IT27W indicators in order to perform routine control of cleaning and thermal disinfection processes in automatic machines.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.  
ISO/TS 15883-5:2005

### Characteristics

Chrome plating SUS304 grade stainless steel re-usable holder of great resistance and durability.

Base: 32 mm wide, 72.5 mm long, pin at the back side.

Basket: 32 mm wide, 37 mm long, 0.5 mm thick.

The holder is designed to mimic the cleaning challenges posed by a large process load during the washing cycle and can be fixed on the tray, which allows reproducible location of the cleaning indicator in the washer-disinfector. When CDWA4 is placed within the holder, half of the test soil is obscured, thus simulating the joint of a surgical instrument, with water stream hitting it indirectly. The other half of the test soil is covered by a mesh, thus posing a difficulty for water stream to reach it, mimicking shadowed or occluded surfaces.

The new design of CDWAH optimizes the control of the washer-disinfector's ability to force water and detergent into and onto the test soil.

### Storage conditions

T = 10-30 °C, RH= 30-80 %, store in dark place.

### Transport conditions

Carry in closed and strengthened boxes in order to avoid damages. The transport of this product does not represent a risk for health.

### Shelf life

500 cycles.

### Packing

1 unit per box.

Packing information: product code and description, manufacturer information, storage conditions, batch number, manufacture date, presentation, bar code and datamatrix code.

### Labelling

Information only on product's box.

### Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

### **Other important information**

Read product's instructions for use thoroughly before use.

### **Precautions**

Store the product within its' original package, away from disinfectants, corrosive or oxidant agents, gases and humidity.

Holder may be hot after running the washing cycle.

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