
Infection Control Division | **Cleaning Indicators**

CDWAH Cleaning Indicator Holder

For Chemdye® CDWA and Integron® IT27W cleaning indicators.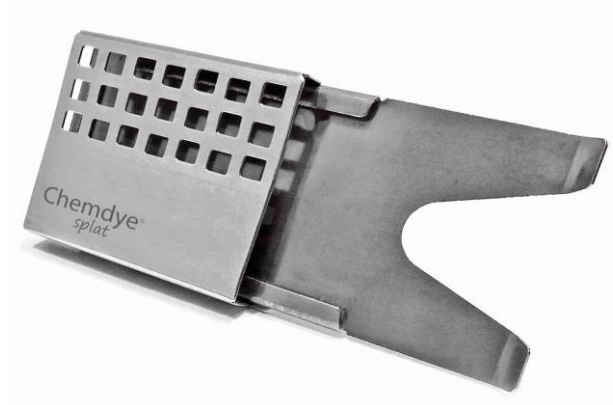

Usage

Designed to hold Chemdye® CDWA and Integron® IT27W indicators in order to perform routine control of cleaning and thermal disinfection processes in automatic machines.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.
ISO/TS 15883-5:2005

Characteristics

Chrome plating SUS304 grade stainless steel re-usable holder of great resistance and durability.

Base: 32 mm wide, 72.5 mm long, pin at the back side.

Basket: 32 mm wide, 37 mm long, 0.5 mm thick.

The holder is designed to mimic the cleaning challenges posed by a large process load during the washing cycle and can be fixed on the tray, which allows reproducible location of the cleaning indicator in the washer-disinfector. When CDWA4 is placed within the holder, half of the test soil is obscured, thus simulating the joint of a surgical instrument, with water stream hitting it indirectly. The other half of the test soil is covered by a mesh, thus posing a difficulty for water stream to reach it, mimicking shadowed or occluded surfaces.

The new design of CDWAH optimizes the control of the washer-disinfector's ability to force water and detergent into and onto the test soil.

Storage conditions

T = 10-30 °C, RH = 30-80 %, store in a dark place.

Transport conditions

It is recommended to respect storage conditions during transport. Transport in closed and strengthened boxes in order to avoid damages.

Transportation of this product does not represent a risk for health.

Shelf life

500 cycles.

Packing


1 unit per box.

Packing information: product code and description, manufacturer information, storage conditions, batch number, manufacture date, presentation, bar code and datamatrix code.

Infection Control Division | **Cleaning Indicators**

CDWAH Cleaning Indicator Holder

For Chemdye® CDWA and Integron® IT27W cleaning indicators.

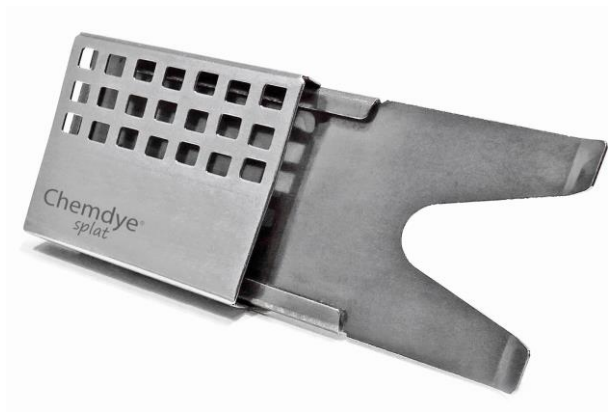Labelling

Information only on product's box.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Store the product within its' original package, away from disinfectants, corrosive or oxidant agents, gases and humidity. Holder may be hot after running the washing cycle.

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