HemoSwab

Sistema de Control de Higiene.

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HemoSwab

Hygiene Monitoring System

Kit for detection of blood residue (hemoglobin) on surfaces

Usage

Designed to detect residual blood (hemoglobin) on surgical instrument surfaces after the cleaning process and prior to the disinfection / sterilization process.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012.

Characteristics

- The kit has a 6.9 cm long high absorption swab, one vial with indicator solution (transparent cap) and one vial with the reaction activator (blue cap). The swab along with the kit's moisturizing solution allows the collection of samples of different surfaces with the same efficiency after the cleaning process.
- Solution volume: 0.5 ml.
- An immediate and visual reading through the appearance of blue-green color on the swab and/or solution indicates the presence of detectable levels of blood. The more blue and stronger the color obtained, the greater the amount of hemoglobin present on the swabbed surface.
- HemoSwab can detect amounts of blood (hemoglobin) as little as 0.1 µg.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

Temperature between 2-25 ° C, 30-80% RH, keep away from light.

Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes to avoid damages.

Transportation of this product does not represent a risk for health.

Shelf-life

18 months

Packaging

24 tests per box (24 vials with indicator solution + 24 vials with reaction activator + 24 swabs) + 2 tubes of moisturizing solution for swabs.

Data on packaging: product code and description, process for which it is used, operating instructions, manufacturer's data.

Labelling

In the package: code and description of the product, lot and expiration date.

Possible target markets

Healthcare and Medical industries.

Other important information (optional)

Read product's directions for use thoroughly before use.

Note: When necessary, limits for each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not open until use. Do not reuse. Protect from light. Do not expose to heat. Do not touch with your hands. Wear gloves, chinstrap and other hygiene measures. Do not freeze.