Infection Control Division | **Cleaning Indicators**

KPRO2-E250 Hygiene Monitoring System

Kit system for detecting residual protein on endoscopes and other cannulated instruments

Usage

Designed to detect proteins on endoscopes and other reusable instruments with hard to reach internal channels or cannulated instruments.

Characteristics

- The system has a flexible high absorption swab SW250, of 2.5 m long and 1.9 mm diameter absorption area as well as a plastic tube with reactive solution. The SW250 swab can be introduced through the channel of the endoscope thus allowing sampling contaminants left after the cleaning process.
- Volume of solution: 0.45 ml.
- An instant and visual reading of a color change indicates the presence of detectable levels of protein. When comparing the color obtained in the solution with the reference color card, a visual estimation of surface cleaning can be made.
- KPRO2-E250 can detect as little as 1 µg of protein.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 2-30 °C, RH = 30-80 %, keep in a dark place.

Transport conditions

Storage conditions should be strictly followed. Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health. The product can be kept at temperature below 30 °C for up to 25 days during transportation without affecting its shelf-life (provided that the subsequent storage conditions are respected).

Shelf life

24 months when stored refrigerated (2-8 °C) and 6 months when stored at room temperature (up to 30 °C).

Infection Control Division | **Cleaning Indicators**

KPRO2-E250 Hygiene Monitoring System

Kit system for detecting residual protein on endoscopes and other cannulated instruments

Packing

5 plastic tubes with reactive solution + 5 SW250 swabs + 2 moisturizer tubes for swabs + 5 registration cards + 1 reference color card.

Packing information: product code and description, storage conditions, manufacturer information and data on label.

Labeling

On each swab's bag: product code and description, batch number and presentation.

On product's box: product code and description, batch number, manufacture and expiration date, presentation, barcode and Data Matrix.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Device Industries.

Other relevant information

Read product's Instructions for Use thoroughly before use.

Precautions

Do not open until ready for use.

Do not reuse.

Do not touch the swab with your hands. Wear gloves, surgical mask and other hygiene measures.

Do not freeze.

Although KPRO2 has high sensitivity, it is not a microbiological monitoring device. Surfaces should not be considered to be sterile based on the results obtained.

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