
Infection Control Division | **Cleaning Indicators**

Lumenia SixFlow LSF1 Cleaning Challenge Kit For Automatic Endoscope Reprocessors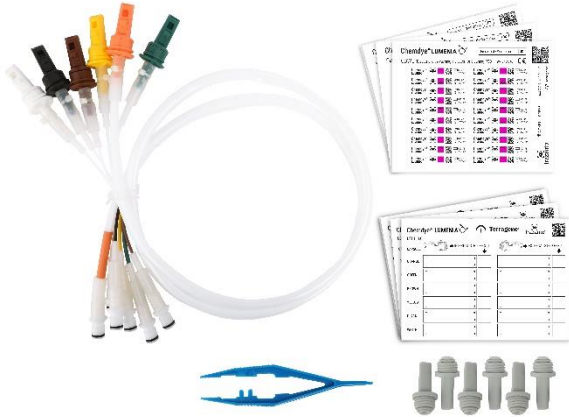

Usage

Challenge Kit designed for monitoring the effectiveness of cleaning performance of flexible endoscopes in Automatic Endoscope Reprocessors.

Applicable regulation

Designed under Quality Management System standard ISO 13485:2016/NS-EN ISO 13485:2016.
ISO 15883-4:2018, ISO/TS 15883-5:2005

Characteristics

Challenge Device

Chemdye® Lumenia SixFlow LSF1 Test Kit consists of a challenge device (Chemdye® Lumenia SixFlow CCDDER61) and cleaning indicators (Chemdye® Lumenia CDWE) for monitoring cleaning performance in Automated Endoscope Reprocessors. The challenge device consists of three polytetrafluoroethylene (PTFE) tubes with an inner diameter of 1 mm, two PTFE tubes with an inner diameter of 2 mm, and a PTFE tube with an inner diameter of 4 mm, all of them 1.5 m long. Each tube has special connectors in one of its ends for plugging it to the Endoscope Reprocessor and a device in the other end that simulates an endoscope's lumen and where the Chemdye® Lumenia CDWE cleaning indicators are placed (one in the inner cavity of the device and the other one in an external slot). Each of the six devices have different colors that match the channels' diameters to be monitored, and are in line with the color scheme that every Reprocessor Machine has in its output terminals. The end of each PTFE tube opposite to that one connected to the device is identified with a colored mark, allowing a fast recognition of the output terminal to which it will be connected. The tubes are joined and protected through a hose that covers them entirely. Chemdye® Lumenia SixFlow LSF1 Test Kit is compatible with Getinge* ED-Flow Automated Endoscope Reprocessor (AER).

Cleaning indicator

Chemdye® Lumenia CDWE cleaning indicator was especially designed for monitoring the performance of cleaning processes. It consists of a test soil, developed in accordance with the requirements described in ISO/TS 15883-5:2005 standard, arranged on a self-adhesive thermolabile synthetic support. The formulation of the cleaning indicator allows the detection of factors that affect the result of the cleaning process.

Size of the indicator strip: 38 mm x 6 mm.

Initial condition: fuchsia color.


Final condition: total elimination of the fuchsia organic substance.

The indicator's ink is not toxic.

*Getinge is a registered trademark of Getinge AB.

Infection Control Division | **Cleaning Indicators**

Lumenia SixFlow LSF1 Cleaning Challenge Kit For Automatic Endoscope Reprocessors

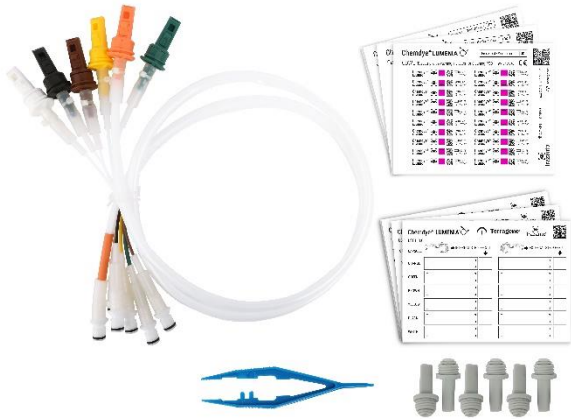NOTE: If unfavorable results are obtained, check quality/quantity of chemicals, water pressure, temperature, cleaning time and general condition of the washing machine.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %. Store in dark place.

Transport conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes in order to avoid damages.

The transport of this product does not represent a risk for health.

Shelf life

Cleaning indicator: 5 years.

Packing

1 Challenge Device + 36 sheets of cleaning indicators (720 indicators) + 6 blockage test capsules + 1 plastic tweezer + 60 Trazanto record cards + Instructions for use.

Packing information: product code and description, intended use, regulation, storage conditions, manufacturer information and data on label.

Labelling

On product's pack: product code, kit batch number and its components' batch numbers, presentation, manufacture and expiration date, barcode and datamatrix code.

Possible target markets

Healthcare and Medical Industries

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Store the product within its original package, away from light, disinfectants, gases and humidity.

Do not re-use the indicator.

Do not touch indicator ink with your hands.

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*Getinge es una marca registrada de Getinge AB.