CD250

Tubos Químicos Multi-variables

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For Dry Heat sterilization control.

Usage

Control of Dry Heat sterilization processes.

Conditions: use in sterilizing ovens at 160 °C for 120 minutes, 170 °C for 60 minutes or 180 °C for 35 minutes.

Applicable Regulation

ISO 11140-1:2014.

Product Classification

Class 1, according to risk.

Authorization

Designed under Quality Management Systems standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Characteristics

Type 4 Multi-variable Chemical Indicator.

Glass ampoule.

Size: 7 mm external diameter x 40 mm length; walls of 0.3 mm thick (approximate measurements).

Volume contained in the ampoule: 0.25 ml.

Extreme identification mark: WHITE SPOT.

Initial color: **red**.

Final color: **green**.

NOTE: As sterilization proceeds, the liquid changes color from red to GREEN. If after treatment, the liquid remains red or amber the materials should be re-sterilized and a fresh tube must be used.

A color guide in the instruction provides an instant visual check of the results.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 2-15°C, RH 30-80%, store in dark place in its original package. Avoid freezing.

Transportation conditions

Transport at a temperature below 30 °C for a maximum of 30 days in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

2 years (when stored under the recommended conditions).

Packing

100 tubes per box.

Packing information: product's description, instructions for use, presentation, regulation, manufacturer information, storage conditions and data on box's label.

Weight per box: 106.1 g.

Labelling

On the label: product's code and description, process for intended use, usage conditions, classification according to regulation, barcode, datamatrix, batch number and expiration date.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's directions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to EO, Steam or any sterilization processes other than Dry Heat.

Do not reuse the sterilizer until the indicator has changed into the color indicated in the product.