CDPCD2X025/P

Helix Process Challenge Device

To evaluate air removal and penetration of sterilizing agent in hollow loads.

Usage

To evaluate air removal and penetration of the sterilizing agent (steam / ethylene oxide / formaldehyde / VH_2O_2) in hollow loads.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

Characteristics

Plastic (polypropylene) capsule of great resistance and durability, to hold a chemical indicator designed for this system. The capsule is connected to a high resistance plastic tube of 0.25 m length and 2 mm internal diameter and connector is coated by silicone protector.

NOTE: Terragene recommends Helix-PCD of 0.25, 0.5, 0.75 or 1.2 m length for VH_2O_2 processes.

Chemical indicators designed for Helix-PCD system: PCD-A-3.5Y / PCD-A-5.3Y / PCD-A-7.0Y / PCD-A-18Y (STEAM); PCD-A-F1 (FORM); PCD-A-E1 (EO); PCD-A-P1 (VH_2O_2).

Environmental conditions during manufacture

T = 15-30 °C, RH 30-80 %.

Storage conditions

T = 10-30 °C, RH 30-80 %, keep out of direct light.

Transport conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Shelf life

250 cycles

Packing

1 Helix-PCD + 1 Cotton bag

Packing information: manufacturer information, storage conditions and data on label.

Labelling

Only on product's box: product code and description, batch number, presentation, barcode and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not place the Helix-PCD device within the cotton bag when sterilizing with VH_2O_2 and Formaldehyde; since it can interfere with these processes.

Do not store the product near sterilizing agents.

Do not expose this product to chemical agents.

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