CDPCD2X075/P

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CDPCD2X075/P

Helix Process Challenge Device

To evaluate air removal and penetration of sterilizing agent in hollow loads.

Usage

To evaluate air removal and penetration of the sterilizing agent (steam / ethylene oxide / formaldehyde / VH_2O_2) in hollow loads.

Conditions:

STEAM: 3.5 min., 134 °C // 5.3 min., 134 °C // 7.0 min., 134 °C // 9.0 min., 134 °C // 18 min., 134 °C.

OE: 240 min., 54 °C, EO 600 mg/l, 60 % RH.

FORM: LTSF processes.

VH_2O_2 : plasma or vaporized Hydrogen Peroxide processes.

Applicable regulation

Less demanding requirements than EN 867-5.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012.

Characteristics

Plastic capsule of great resistance and durability, to hold a chemical indicator designed for this system. The capsule is connected to a high resistance plastic tube of 0.75 m length and 2 mm internal diameter. Special silicone connector coated by external connector.

NOTE: Terragene recommends Helix-PCD of 0.25, 0.5, 0.75, or 1.2 m length for VH_2O_2 processes.

Chemical indicators designed for Helix-PCD system: PCD-A-3.5 / PCD-A-5.3 / PCD-A-7.0 / PCD-A-9.0 / PCD-A-18 (STEAM); PCD-A-F1 (FORM); PCD-A-E1 (EO); PCD-A-P1 (VH_2O_2).

NOTE: If 4 bars change to the final color, the result is successful, the sterilizer works properly. If one or more bars have not turned to the final color, this shows a failure in sterilizing agent penetration, therefore an equipment malfunction.

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 10-30 °C, RH 30-80 %, keep out of direct light.

Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Shelf-life

Helix-PCD: 250 cycles

Packing

1 Helix-PCD + 1 Cotton bag

Packing information: product code and description, manufacturer information and data on label.

Labelling

Only on product's box: product code and batch number.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information (optional)

Read product's directions for use thoroughly before use.

Note: When necessary, limits of each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not place the Helix-PCD device within the cotton bag when sterilizing with VH_2O_2 and Formaldehyde; since it can interfere with these processes.

Do not store the product near sterilizing agents.

Do not expose this product to chemical agents.