Infection Control Division | **Chemical Indicators**

## IT26-1YS Unique point Integrator Indicator For Steam sterilization processes

### Usage

#### United States

The integrator Terragene Integron® IT26-1YS is designed to chemically react over time with the critical parameters of steam sterilization cycle within a specified tolerance. The integrating indicator strip is intended to be placed in each pack, pouch, container, tray or other containment device to function as an independent monitor of critical parameters for the following sterilization cycles: Gravity Displacement: 121 °C for 30 minutes, 132 °C for 15 minutes, 132 °C for 25 minutes, 135 °C for 10 minutes; Dynamic Air Removal (Vacuum Assist): 132 °C for 4 minutes, 135 °C for 3 minutes. SV121 °C/16.5 min., SV132 °C/2.0 min, SV135 °C/1.2 min.

#### Outside the United States

Control of Steam sterilization processes between 121 °C and 135 °C.

#### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016. ISO 11140-1:2014.

#### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

#### Classification

Class 1, according to risk (ANMAT).

#### FDA 510(k)

K191021.

#### Characteristics

Type 5 Integrating Indicator (according to ISO 11140-1).

One-side-laminated strips printed with indicator ink.

Size: 70 mm x 23 mm.

Initial color: **yellow**.

Final color: **black**.

100 % Toxic Heavy Metal free.

Infection Control Division | **Chemical Indicators**

## IT26-1YS Unique point Integrator Indicator For Steam sterilization processes

Integration condition calibrated with the kill time of a  $10^6$  *Geobacillus stearothermophilus* ATCC®\* 7953 population, calculated in BIER (Biological Indicator Evaluator Resistometer). Conditions: saturated steam at 121 °C, 128 °C, 135 °C.

The product ensures an adequate control of the efficacy of sterilization processes (temperature, time, quality of steam).

To interpret results visually, use the reference found in the indicator. Optionally, the readout of the chemical indicator can be done using Trazanto Automatic Quality Control and Traceability System (see instructions for use in device's manual).

### **Environmental conditions during manufacture**

T = 15-30 °C, RH = 30-80 %.

### **Storage conditions**

T = 10-30 °C, RH = 30-80 %, keep in a dark place.

### **Transport conditions**

It is recommended to store it away from sunlight and at a temperature between 10 - 30 °C.

Products should be transported in closed and reinforced boxes in order to avoid damages.

The transport of this product does not represent any risk for human health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

### **Shelf life**

5 years.

### **Packing**

200 or 500 units per aluminum foil bag.

Packing information: product description, instructions for use, regulation, storage conditions, manufacturer information and data on label.

### **Labelling**

On product's packing: product code and description, process for intended use, classification according to regulation, color change, presentation, batch number, expiration date, bar code and datamatrix code.

### **Possible target markets**

Healthcare, Food, Pharmaceutical and Medical Industries.

\*ATCC® is a registered trademark of American Type Culture Collection.

Infection Control Division | Chemical Indicators

## IT26-1YS Unique point Integrator Indicator For Steam sterilization processes

### Other important information

Read product's instructions for use thoroughly before use.

### Precautions

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization processes other than Steam.

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