
# IT27-7YS

## Unique Point Emulator Indicator

For Steam sterilization processes.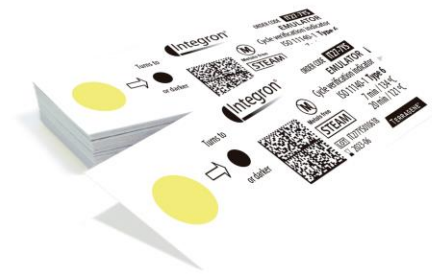

### Usage

Control of Steam sterilization processes.

**Emulation conditions:** 7minutes, 134 °C // 20 minutes, 121 °C.

### Applicable Regulation

Designed under Quality Management Systems standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11140-1:2014.

### Classification

Class 1, according to risk.

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

### Characteristics

Type 6 Emulator Indicator.

One-side-laminated strips printed with indicator ink.

Size: 72.0 mm x 23.0 mm.

Initial color: **yellow**.

Final color: **black**.

The product ensures an adequate control of the efficacy of sterilization processes (temperature, time, quality of steam).

100 % Toxic Heavy Metals free

### Environmental conditions during manufacture

T= 15-30 °C, RH= 30-80 %.

### Storage conditions

T = 10-30 °C, RH= 30-80 %, keep out of direct light.

### Transportation conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

### Shelf-life

5 years.

**Packing**

200 units per aluminum foil bag.

Packing information: product description, process for intended use, instructions for use, regulation, storage conditions, manufacturer information and data on label.

**Labelling**

On product's packing: product code and description, process for intended use, use conditions, classification according to regulation, color change, presentation, batch number, manufacture and expiration date, bar code and datamatrix code.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical Industries.

**Other important information**

Read product's directions for use thoroughly before use.

**Description of identified and/or predictable risks which could represent a risk for the product**

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization processes other than Steam.

Do not reuse the sterilizer until the indicator has changed into the color indicated in the product.

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