
Infection Control Division | Chemical Indicators

## KBD8948X/1 Bowie-Dick Test Card Kit

Includes Bowie-Dick Test Card Chemical Indicator and its Holder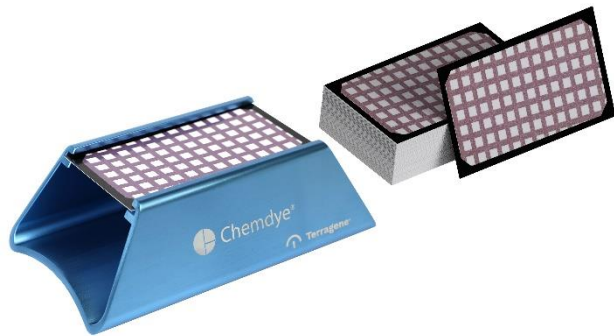

### Usage

Terragene® Chemdye® Bowie-Dick Test Card Kit KBD8948X/1 is designed for testing air removal efficiency for cycles of 132 °C for 4 minutes and 134 °C for 3.5 minutes in dynamic-air-removal steam sterilizers.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016, ISO 11140-1:2014 and ISO 11140-5:2007.

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

### Classification

Class 1, according to risk (ANMAT).

### FDA 510(k)

K200272.

### Characteristics

#### BD8948X/1

Bowie-Dick Test Card Chemical Indicator.

Type 2 Chemical Indicator.

Initial Color: **purple**.

Final Color: **green**.

Size: 89 mm x 48 mm x 0.33 mm

Material: one-side-laminated cardboard.

Cycle information can be recorded on the back of the card.

**NOTE:** Air removal problems are shown as heterogeneous color change, i.e. the presence of purple color.

**100 % Toxic Heavy Metal free**

#### BD8948H

Stainless steel re-usable Holder for Bowie-Dick Test Card of great resistance and durability coated by a protective layer.

Keeps BD8948X/1 Test Card in place for proper assessment of sterilization cycle.

Height: 42.7 mm.


Base: 133.2 mm x 72.4 mm.

Upper part where the BD8948X/1 Test Card fits: 93 mm x 53.5 mm.

Infection Control Division | **Chemical Indicators**

## KBD8948X/1 Bowie-Dick Test Card Kit

Includes Bowie-Dick Test Card Chemical Indicator and its Holder

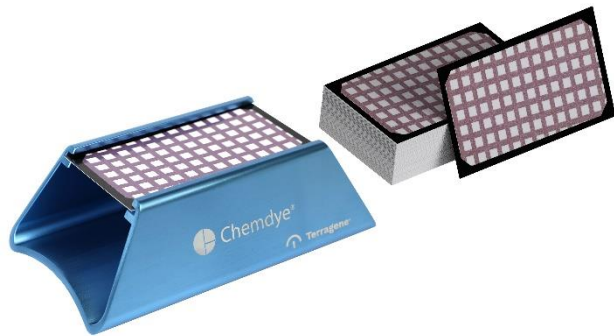### Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

### Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

### Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10 - 30 °C.

Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

### Shelf life

5 years.

### Packing

1 Holder + 120 Bowie-Dick Test Cards per box.

Packing information: product description, process for intended use, manufacturer information, storage conditions and data on the label.

### Labelling

On product's packing: Product code, classification according to regulation, color change, presentation, batch number, manufacture date, expiration date, barcode and datamatrix code.

### Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

### Other important information

Read product's instructions for use thoroughly before use.

### Precautions

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization process other than Steam.

Do not use a Bowie-Dick Test Card in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase) or any kind of sterilization processes other than vacuum-assisted Steam.

Do not use the sterilizer until the indicator has reached its homogeneous endpoint color.

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