Infection Control Division | **Chemical Indicators**

KBD8948X Bowie-Dick Test Card Kit

Includes Bowie-Dick Test Card Chemical Indicator and its Holder

Usage

Bowie-Dick test monitors the effectiveness of air removal in vacuum-assisted steam sterilizers. Designed for cycles of 3.5 minutes at 134 °C and 4 minutes at 132 °C.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/ EN ISO 13485:2016.
ISO 11140-1:2014 and ISO 11140-4:2007.

Authorization

ANMAT (Argentinian National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

Characteristics

BD8948X

Bowie-Dick Test Card Chemical Indicator.

Type 2 Chemical Indicator.

Initial Color: **purple**.

Final Color: **green**.

Size: 89 mm x 48 mm x 0.33 mm

Material: one-side-laminated cardboard.

Cycle information can be recorded on the back of the card.

NOTE: Air removal problems are shown as heterogeneous color change, i.e. the presence of purple color.

100 % Toxic heavy metal free

BD8948H

Blue anodized aluminum re-usable holder for Bowie-Dick Test Card.

Height: 42.7 mm

Base: 133.2 mm x 72.4 mm

Upper part where the BD8948X Test Card fits: 93 mm x 53.5 mm.

Environmental conditions during manufacture

T= 15-30 °C, RH= 30-80 %.

Storage conditions

T= 10-30 °C, RH= 30-80 %, keep away from sunlight. It is recommended to protect the indicator Test Card after use inside a plastic foil for its storage.

Infection Control Division | Chemical Indicators

KBD8948X Bowie-Dick Test Card Kit

Includes Bowie-Dick Test Card Chemical Indicator and its Holder

Transport conditions

It is recommended to respect the storage conditions during transport.

Products should be transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Shelf life

5 years.

Packing

1 Holder + 120 Bowie-Dick Test Cards per box.

Packing information: Product code and description, process for intended use, storage conditions and manufacturer information.

Labelling

On product's packing: Product code, regulatory and standards information, color change, presentation, batch number, manufacture and expiration date, barcode and Data Matrix.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Devices Industries.

Other important information

Read product's Instructions for Use thoroughly before use.

Precautions

Do not use a Bowie-Dick Test Card in gravity displacement steam sterilization cycles (where there is no pre-vacuum phase) or any kind of sterilization processes other than vacuum-assisted steam.

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