MK50

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MK50

Marker with water based indicator ink

For Formaldehyde sterilization processes.

Usage

Designed for permanent reliable marking of sterilization packaging material, or to write on labels or tapes adhered to them. For Formaldehyde sterilization processes.

Applicable Regulation

ISO 11140-1:2014

Classification

Class 1, according to risk.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Technology) PM 1614-4.

Characteristics

Type 1 Chemical Indicator.

The marker enables a smooth writing to avoid damaging the packing material.

Dimensions: 14 cm length x 1.7 cm diameter.

No domestic uses.

Initial color: **bordeaux**

Final color: **green**

100 % Toxic Heavy Metals free

Environmental conditions during manufacture

15-30 °C (room temperature), RH 30-80 %.

Storage conditions

T = 10-25 °C, RH 30-80 %, keep out of direct light.

Transportation conditions

Storage conditions should be strictly followed. Carry in closed and strengthened boxes or bags. Transportation of this product does not represent a risk for health.

Shelf-life

12 months.

Packing

1 marker per plastic bag.

Labelling

On product's bag: product code and description, process for intended use, classification according to regulation, batch number and expiration date.

On product: product code and description, process for intended use, classification according to regulation, warnings and directions for use.

Possible target markets

Healthcare and Industry.

Other important information (optional)

Read product's directions for use thoroughly before use.

NOTE: When necessary, limits of each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to Steam, EO or any sterilization processes other than Formaldehyde.

Do not reuse the sterilizer until the ink has changed into the color indicated in the product.