# PCD-A-P1

## Test strip with chemical indicator

For use in Helix-PCD system.

### Usage

Chemical Indicators designed to be used in the Helix-PCD system for control of plasma or vaporized Hydrogen Peroxide sterilization processes.

### Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.  
ISO 11140-1:2014.

### Product Classification

Class 1, according to risk.

### Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

### Characteristics

Type 2 Chemical Indicator.

Self-adhesive varnished strip printed with indicator ink.

Size: 76 x 6 mm.

10 strips per sheet. Size: 80 x 98.5 mm.

Initial color: **purple**

Final color: **green**

100 % Toxic Heavy Metal free

NOTE: If 4 bars change to the final color, the result is successful, the sterilizer works properly. If one or more bars have not turned to the final color, this shows a failure in sterilizing agent penetration, therefore an equipment malfunction.

### Environmental conditions during manufacture

T = 15-30 °C, RH 30-80 %.

### Storage conditions

T = 10-30 °C, RH 30-80 %, keep out of direct light.

### Transport conditions

Storage conditions should be strictly followed. Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

### Shelf life

5 years

### Packing

100 strips in sheets of 10 units.

Packing information: product description, classification according to regulation, storage conditions, manufacturer information and data on label.

**Labelling**

On product packing: product code, process for intended use, presentation, color change, batch number, expiration date, barcode and datamatrix code.

**Possible target markets**

Healthcare, Food, Pharmaceutical and Medical industries.

**Other important information**

Read product's instructions for use thoroughly before use.

**Precautions**

Do not store the product near sterilizing agents.

Do not expose this product to Steam, Dry Heat, Formaldehyde or any sterilization process other than H<sub>2</sub>O<sub>2</sub>.

Do not reuse the sterilizer until the color change of the chemical indicator is the one indicated in the product.

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